

INSTRUCTIONS FOR USE

PRECAUTIONS:

General

- These devices are intended only to be used in a hospital environment by suitably qualified medical professionals with training and knowledge of the indications and techniques required for urodynamic procedures.
- The devices are supplied with standard luer fittings. When connecting the luer to the urodynamic machinery ensure the fitment is secure (do not over tighten as this may damage the luer) prior to commencing the urodynamic procedure.
- Only insert the minimum required length of catheter for the procedure to ensure the catheter doesn't become knotted.
- Before use, inspect the devices for any evidence of damage. Do not use if there is visible damage.
- Should balloon rupture occur during use, care should be taken to ensure that all balloon fragments have been removed from the patient.
- On completion of the examination, dispose of the devices in accordance with local clinical disposal procedures.

STORAGE INSTRUCTIONS:

- Store at room temperature and avoid exposure to excessive light, heat, and damp conditions.

MANUFACTURER:

Refer to product label below for manufacturer and product details.



Blue Box Medical Ltd
Unit 29 New Forest Enterprise Centre
Rushington Business Park Chapel Lane
Totton, Southampton, SO40 9LA UK
Tel 02380 669000 / Fax 02380 669002

info@blueboxmedical.co.uk

REF DBS-1800


DOUBLE SPIKE UROLOGY IRRIGATION SET, 1.8m

CONTENTS 20

LOT 16744/1


STERILE EO

 2021-06

 2026-06



EC REP Advena Ltd.
Tower Business Centre 2nd Flr
Tower Street, Swatar
Rev 6.0 BKR 4013 Malta

 **PB Medical Ltd.**
Unit 10, Northbrook Close
Worcester, WR3 8EP
United Kingdom

CE
1639