

en Instructions for Use

Surgical Skin Marker with Ruler formulated for ChloroPrep™ Products



INDICATIONS FOR USE:
Skin marker is used for surgical marking on intact skin.

INSTRUCTIONS FOR USE:

1. For sterile pen only. Remove sterile pen from packaging.
2. Remove the pen cap to expose the pen tip.
3. Mark desired areas on skin with the skin marker according to facility protocol.
4. Allow the markings to dry on the skin for approximately 1 minute before rubbing or applying prep solution.
5. Apply prep solution as instructed by prep solution manufacturer.

PRECAUTIONS:

- Formulated specifically for ChloroPrep™ products. Also works well with other popular prep solutions.
- It is important the skin is completely dry prior to applying the skin marker ink. The skin marker ink will not adhere to prep solution or wet skin.
- Cleaning the skin with isopropyl alcohol before marking may increase the adherence of the ink to the skin.
- When prep solution is applied over the skin marker ink, some of the marking will smear, but the skin marker ink will still be visible.
- This device is single use only. Reuse of this device may lead to patient cross contamination and/or device failure.
- Rulers packaged with skin markers are not measuring devices and should be used as an indicator only and not for accurate measurements.
- Not for ocular use.
- Not for use on infants.
- Device is intended for limited contact duration. Less than or equal to 24 hours.

REF 2710



2797



REF 2710NS



SYMBOL LEGEND:

- Catalogue Number
- Batch Code
- Use By Date
- Manufacturer
- Consult Instructions For Use
- Non-Sterile
- Do Not Re-Use
- Do not use if package is opened or damaged
- Sterilized using irradiation
- Not made with natural rubber latex
- Date of Manufacture
- Rx ONLY**

Distributed By:
Becton, Dickinson and Company

Becton Dickinson Pty Ltd
66 Waterloo Rd,
Macquarie Park NSW 2113
Australia

Becton Dickinson Limited
14B George Bourke Drive
Mt. Wellington, Auckland
1060 New Zealand

BD FR Rev. 2 2019/07 01

Aspen Surgical Products, Inc.

6945 Southbelt Dr. SE
Caledonia, MI 49316 USA
Phone 616-698-7100
Toll-Free 888-364-7004
www.aspensurgical.com

Hill Rom SAS
Zone Industrielle de Talhouet
56330 Pluvigner, France

BD, the BD Logo and BD ChloroPrep are trademarks of Becton, Dickinson and Company. ©2019 BD and its subsidiaries. All rights reserved. www.bd.com