# Safeflow Extension Set

<b>GB</b>	Instructions for use	<b>6</b>
<b>DE</b>	Gebrauchsanweisung	<b>6</b>
<b>BG</b>	Инструкция за употреба	<b>7</b>
<b>CN</b>	使用说明	<b>7</b>
<b>CZ</b>	Návod k použití	<b>8</b>
<b>DK</b>	Brugsanvisning	<b>8</b>
<b>EE</b>	Kasutusjuhised	<b>9</b>
<b>ES</b>	Instrucciones de uso	<b>9</b>
<b>FI</b>	Käyttöohje	<b>10</b>
<b>FR</b>	Mode d'emploi	<b>10</b>
<b>GR</b>	Οδηγίες χρήσης	<b>11</b>
<b>HR</b>	Upute za upotrebu	<b>11</b>
<b>HU</b>	Használati utasítás	<b>12</b>
<b>IT</b>	Istruzioni per l'uso	<b>12</b>
<b>LT</b>	Naudojimo instrukcija	<b>13</b>
<b>LV</b>	Lietošanas instrukcija	<b>13</b>
<b>NL</b>	Gebruiksaanwijzing	<b>14</b>
<b>NO</b>	Bruksanvisning	<b>14</b>
<b>PL</b>	Instrukcja użytkowania	<b>15</b>
<b>PT</b>	Instruções de utilização	<b>15</b>
<b>RO</b>	Instrucțiuni de utilizare	<b>16</b>
<b>RS</b>	Uputstvo za upotrebu	<b>16</b>
<b>RU</b>	Инструкция по применению	<b>17</b>
<b>SE</b>	Bruksanvisning	<b>17</b>
<b>SI</b>	Navodila za uporabo	<b>18</b>
<b>SK</b>	Návod na použitie	<b>18</b>
<b>TH</b>	คำแนะนำในการใช้งาน	<b>19</b>
<b>TR</b>	Kullanım Kılavuzu	<b>19</b>
<b>VN</b>	Hướng dẫn sử dụng	<b>20</b>

**B | BRAUN**

B. Braun Melsungen AG  
34209 Melsungen  
Germany

[www.bbraun.com](http://www.bbraun.com)

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## GB Instructions for use

Smallbore extension set for infusion lines.

The extension line allows manipulation away from injection site, which ensures treatment of patient by infusion, transfusion, aspiration of blood samples and intermittent injection without the use of needles.

Ref. code	Description	Materials used
4097154N	Smallbore Extension Set with Safeflow valve	SI, PC, PUR, MABS, PE latex-free, DEHP-free, PVC-free
4097145N	Smallbore Y-Extension Set with two Safeflow valves	SI, PC, PUR, PE, MABS, PVC-U latex-free, DEHP-free
4097148N	Smallbore Y-Extension Set with two Safeflow valves and back check valve	SI, PC, PUR, PE, MABS, PVC-U, SAN, latex-free, DEHP-free

### Use Aseptic Technique

Swab injection site prior to use per institutional protocol. Use a male luer fitting without a needle. Luer slip syringes should be hand-tightened into the valve using a 90° turn. Flush and replace per national and/or institutional protocol. Disinfect the valve with a Softa® Cloth\*) wipe or acc. to its institutional protocol.

\*) Recommended disinfectant: Softa® Cloth (B. Braun Medical AG, Sempach)

### Warning

Re-use of single-use devices creates a potential risk of patient or user. It may lead to contamination and/or impairment of functional capability. Contamination and/or limited functionality of the device may lead to injury, illness or death of the patient.

Do not use if packaging is damaged.  
Do not resterilize.

**CAUTION! Do not leave connected luer slip syringes unattended. Close clamp before disconnecting the syringe.**

Turn the syringe counter-clockwise to remove.

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**CO** Importado por B. Braun Medical S.A.