

148x210mm



Super Absorbent Dressing Backed Non-adhesive Sterile

PRODUCT CODES

537840ZD, 537841ZD, 537842ZD, 537843ZD & 537863ZD

Description

Super Absorbent Dressing is a sterile, non-adherent, absorbent dressing with high absorbency properties and used on moderate to severe exudation. Consists of a soft wound contact layer, a multi-layer absorbent core and a fluid-repellent blue backing, used as a primary or secondary dressing to absorb wound exudate and provide an optimal wound healing environment.

Indication

Pressure ulcers
Leg and foot ulcers
Post-operative wounds
Superficial and partial thickness burns

Instructions For Use

1. Clean the wound according to the practice method
2. Select a dressing which is large enough to provide a 2-3 cm contact edge, to protect the surrounding skin from impregnation
3. Open the pouch using an aseptic technique and take out the product
4. Apply the white side of the super absorbent dressing to the wound
5. Fixate the Super Absorbent dressing with adhesive tape or bandage
6. Check wound and dressing regularly. Dressings can be used for up to 7 days, if the dressing becomes saturated with exudate, should be replaced immediately



0123



LOT SEE PACKAGING

Rociale Healthcare Ltd, Cwm Cynon Business Park (North),
Mountain Ash, Rhondda Cynon Taf, South Wales, CF45 4ER, UK

Zhende Medical Co., Ltd.
Gaobu Town, 312035, Shaoxing, Zhejiang, China.
www.zhende.com

Shanghai International Holding Corp. GmbH(Europe)
Eiffestraße 80 20537 Hamburg GERMANY

01/22 V1

■ 285
■ 285 60%