



Before using an oral swab...

Please check the head is firmly attached to the paper stick.


Rociale Healthcare is committed to promoting patient safety and welfare. Any object placed into the mouth could present a choking hazard. Therefore as an added precaution please follow the instructions below carefully.

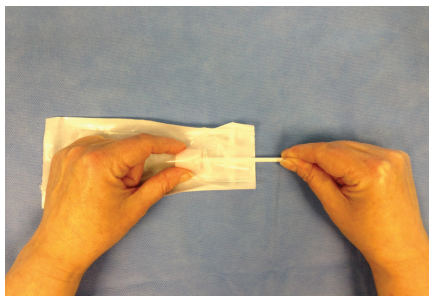

Grip the foam head between the packaging and gently tug on the paper stick.

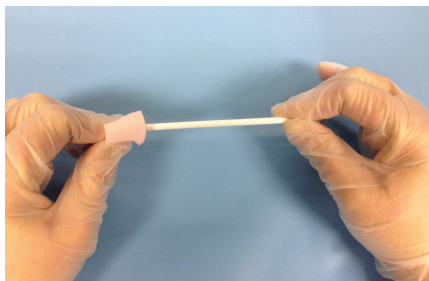

Alternatively, use a sterile gloved hand and check the foam head is attached to the paper stick.

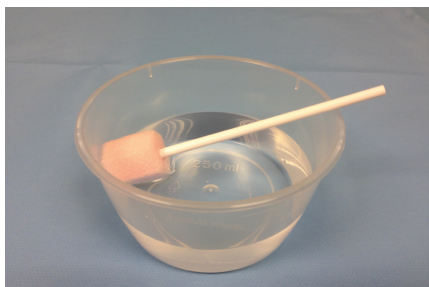

DO NOT allow the oral swab to soak in liquid as the paper stick will soften over time and the foam head is liable to become detached.

After use, discard the swab immediately.

If you break down and distribute these packs, please ensure this information is passed on.



Rociale Healthcare Ltd, Ty Mynydd, Cwm Cynon Business Park (North), Mountain Ash, Rhondda Cynon Taff, South Wales, CF45 4ER, UK



Advena Limited Tower business park, 2nd Flr, Tower Street, Swatar, BKR 4013, Malta

Call: 01443 471300

E-mail: info@rociallehealthcare.com

Web: www.rociallehealthcare.com

