

OBSERVE UNIVERSAL PRECAUTIONS ON ALL PATIENTS

REPORT NEEDLE STICKS IMMEDIATELY AND FOLLOW ESTABLISHED PROTOCOL.

- Percutaneous puncture with a contaminated needle may lead to serious illness such as hepatitis, HIV (AIDS), or other infectious diseases.

- Resheathing needles is hazardous.

NON-PYROGENIC - STERILE:

Unless package has been opened or damaged.

DO NOT AUTOCLAVE

As with any spinal procedure, to help avoid needle breakage, DO NOT ATTEMPT TO STRAIGHTEN A BENT NEEDLE.

Care should be taken to avoid needle damage. Repeated repositioning may increase the risk of needle breakage. Observe needle carefully during procedures.

Remove spinal needle and introducer needle together. Replace with a new needle.


ENSURE SYRINGE IS SEATED FIRMLY IN NEEDLE HUB.

To be used by properly qualified and trained health professionals.

CAUTION: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

Re-use may lead to infection or other illness/injury.

BD Spinal Needle

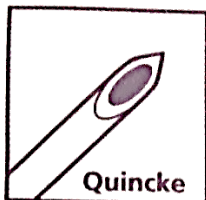


- SPINAL NEEDLE Quincke Type Point
- AGUJA ESPINAL, punta tipo Quincke
- Agulha espinal de ponta Quincke
- AIGUILLE SPINALE Biseau de Quincke
- SPINALKANÜLE mit Quinckeschliff
- AGO SPINALE:Punta tipo Quincke
- Spinale naald met Quincke punt
- SPINALNÄL med Quincke slipad spets
- SPINALKANYLE med Quincke slib
- SPINAALINEULA, Quincke tyypinen kärki
- СПИНАЛЬНАЯ ИГЛА с заточкой типа Quincke
- ΒΕΛΟΝΑ ΟΣΦΥΝΟΤΙΑΙΑΣ ΠΑΡΑΚΕΝΤΗΣΗΣ με Άκρο τύπου Quincke
- Igła podrażczynówkowa z końcówką typu Quincke point
- Spinál tû Quincke típusú hegyyel
- SPINÁLNÍ JEHLA s Quincke hrotem
- Spinálna ihla s hrotom typu Quincke
- SPINALNA IGLA s konico tipa Quincke
- SPINAALNÖEL Quincke-tüüpi otsikuga
- SPINALINÉS ADATOS Quincke tipo galiukas
- SPINÁLÁ ADATA ar Quincke tipa smaili
- SPINALNÄL med spiss av Quincke-type
- SPINAL İĞNE Quincke Tipi Uç
- إبرة شوكية من نوع كوينكه نقطة
- СПИНАЛНА ИГЛА със скосен връх тип Quincke
- Spinalna igla s Quincke vrhom
- Квинке типті спинальді ине
- Ac spinal cu vârful tip Quincke
- Spinalna igla sa vrhom „Quincke Point“
- Спинална голка з заточкою типу Квінке

BD Whitacre Needle




- WHITACRE Pencil Point Spinal Needle
- Aguja espinal WHITACRE punta lápiz
- Agulha espinal de ponta de lápis WHITACRE
- Aiguille spinale WHITACRE, pointe crayon
- WHITACRE Spinalkanüle mit Bleis-tiftspitze
- Ago spinale WHITACRE con punta a matita
- WHITACRE spinale naald met potloodpunt
- WHITACRE spinalnäl med pencil point spets
- WHITACRE spinalkanyle med pencil point slib
- WHITACRE spinaalineula, kynätyypinen kärki
- Спинальная игла с карандашной заточкой типа Whitacre
- Βελόνα Οσφυονωτιαίας Παρακέντησης τύπου Whitacre Pencil Point
- Igła podrażczynówkowa typu WHITACRE z końcówką pencil point
- Spinál tû ceruza végű hegyyel Whitacre típus
- Spinální jehla k tužkovým hrotem Whitacre
- Spinálna ihla s tuzkovým hrotom Whitacre
- Spinalna igla Whitacre s konico v oblíki svinčnika
- Whitacre Pencil-Point-spinaalnöel
- Spinaliné adata su Whitacre „pieštuko“ tipo galiuku
- Whitacre spinälá adata ar žimūļa formas smaili
- WHITACRE spinalnäl med blyantspiss / Pencil point
- WHITACRE Kalem Uçlu Spinal İğne
- الإبرة الشوكية القلمية من وايتكير
- Спинална игла с моливоводобен връх тип WHITACRE
- Whitacre spinalna igla s pencil point vrhom
- Whitacre қарындаш типті спинальді инеси
- Ac spinal de tip pencil point Whitacre
- Spinalna igla Whitacre „Pencil Point“
- Спинална голка з заточкою типу олівець Whitacre

BD Neonatal

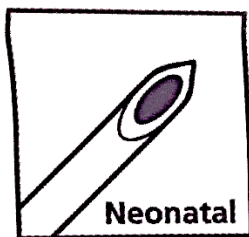


- Neonatal Lumbar puncture needle
- Aguja para punción lumbar neonatal
- Agulha de punção lombar neonatal
- Aiguille de ponction lombaire pour néonatalogie
- Neonatal-Lumbalpunktionskanüle
- Ago per puntura lombare neonatale
- Naald voor neonatale lumbaal-punctie
- Neonatal lumbalpunktionsnål
- Neonatal lumbalpunkturkanyle
- Neonataalilumbaalineula
- Игла для поясничной пункции у новорожденных
- Βελόνα οσφυϊκής παρακέντησης νεογνών
- Iгла do nakłuc' lędźwiowych u noworodków
- Lumbálpunkció-tű újszülöttekhez
- Jehla pro neonatální lumbální punkci
- Ihla pre neonatálnu lumbálnu punkciu
- gla za lumbalno punkcijo novorojenčkov
- Neonataalse lumbaarpunktsiooni nõel
- Naujagimių juosmeninės punkcijos adata
- Neonatālā lumbārās punkcijas adata
- Nål for neonatal lumbalpunksjon
- Neonatal Lumber ponksiyon iğnesi
- إبرة البزل القطني للرضع
- Игла за лумбална пункция при новородени
- Iгла za neonatalnu lumbalnu punkciju
- Неонатальдік люмбальді пункцияға арналған ине
- Ac pentru puncție lombară la nou născuți
- Iгла za lumbalnu punkciju za novorođenčad
- Голка для люмбальной пункции новорожденных

BD Spinal Needle

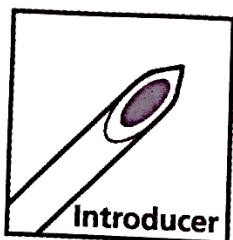


- Spinal Needle Introducer
- Introducator de aguja espinal
- Introducator de Agulha Espinal
- Introducteur pour Aiguille
- Führungskanüle für Spinal
- Introduttore per Ago Spinale
- Introducer voor spinale
- Införare till spinalnål
- Introducer til spinalkanyle
- Spinaalineulan ohjainneula
- Интродьюсер для спинальной
- Εισαγωγέας Βελόνας Οσφυονωτιαίας Παρακέντησης
- Prowadnica do igły podparęczynówkowej
- Gerinctű bevezető
- Zavaděč spinální jehly
- Zavadzac spinálnej ihly
- Vodilo subarahnoidne igle
- Spinaalnõela juhtnõel
- Spinalinés adatatos intubatorius
- Spinālās adatas ievadītājs
- Innføring for spinalnål
- Spinal iğne Introduseri
- إدخال إبرة شوكية
- Интродюсер за спинална игла
- Uvodnik za spinalnu iglu
- Спинальді өткізгіш ине
- Dispozitiv de introducere pentru ac spinal
- Uvodnik za spinalnu iglu
- Интродюсер для спинальной голки

BD Spinal Needle

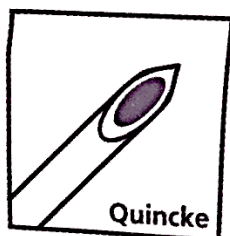


+

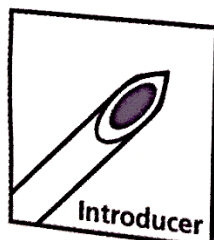


- Spinal Needle with Introducer
- Aguja Espinal con Introducator
- Agulha Espinal com Introducator
- Aiguille Spinale avec Introducteur
- Spinalkanüle mit Führungskanüle
- Ago Spinale con Introduttore
- Spinale Naald met Introducer
- Spinalnål med införare
- Spinalkanyle med Introducer
- Spinaalineula ohjainneulalla
- Игла для спинномозговой Пункции с направляющей
- Βελόνα οσφυονωτιαίας Παρακέντησης με εισαγωγέα
- Iгла podparęczynówkowa z igłą prowadzącą
- Spináltű vezetőtűvel
- Spinální jehla se zavaděčem
- Spinalna ihla so zavadzačom
- Spinalna igla z vodilom
- Spinaalnõel koos sisestusseadmega
- Spinalinè adata su įvedikliu
- Muguras smadzeņu punkcijas adata ar ievadītāju
- Spinalnål med innføringsshylse
- Introdüserli Spinal iğne
- إبرة نخاعية مع مدخال
- Спинална игла с интродюсер
- Spinalna igla s uvodnikom
- Жұлын пункциясына арналған, интродьюсермен жиынтықталған ине
- Ac spinal cu dispozitiv de introducere
- Spinalna igla sa uvodnikom
- Спинальна голка з интродюсером

BD Whitacre Needle

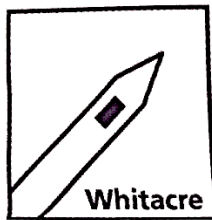

+

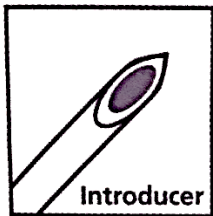


- Whitacre Pencil Point Spinal Needle with Introducer
- Aguja espinal Whitacre punta lápiz con Introducutor
- Agulha espinal de ponta de lápis Whitacre com Introducutor
- Aiguille spinale Whitacre, pointe crayon avec Introducuteur
- Whitacre Spinalkanüle mit Bleistiftspitze mit Führungskanüle
- Ago Spinale Whitacre con punta a matita e Introducuttore
- Whitacre spinale naald met potloodpunt met Introducer
- Whitacre spinalnål med pencil point spets med Införare
- WHITACRE spinalkanyle med pencil point slip med introducer
- WHITACRE spinaalineula, kynätyyppinen kärki Ohjainneulalla
- Спинальная игла с карандашной за точкой типа Whitacre с направляющей
- Βελόνα οσφουονωτιαίας οπονδουλκής Παρακ ντησης στήλης τύπου Whitacre Pencil point με εισαγωγή
- Iгла podrajeczynówkowa z końcówką typu Pencil Point z ostrzem Whitacre i igłą prowadzącą
- Whitacre „ceruzahegyű” spináltű vezetőtűvel
- Atraumatická spinální jehla Whitacre se zavaděčem
- Spinálna ihla s guľatou koncovkou a zavádzačom Whitacre
- Spinalna igla z vodilom Whitacre Pencil Point
- Whitacre'i pencil-point-tipuga spinaalnõel koos sisestusseadmega
- „Whitacre” smailejančios formas spinalinė adata su įvedikliu
- Whitacre konusveida muguras smadzeņu punkcijas adata ar ievadītāju
- Whitacre blyantspiss spinalnål med innføringshylse
- Introdüserli Whitacre Kalem Uçlu Spinal İğne
- إبرة نخاعية ذات رأس حاد مع مدخال من ويتاكر
- Спинална игла с моливодоодобен връх тип Whitacre и интродюсер
- Whitacre spinalna igla s pencil point vrhom i uvodnikom
- Жұлын пункциясына арналған, интродьюсермен жиынтықталған, қарындаш типті Whitacre инесі
- Ac spinal de tip pencil point Whitacre cu dispozitiv de introducere
- Whitacre spinalna igla sa Pencil Point vrhom i uvodnikom
- Спинальна голка з заточкою типу олівець Whitacre з інтродюсером



STERILE EO

C € 0318

 Becton Dickinson S.A., Camino de Valdeoliva, s/n,
28750 San Agustín del Guadalix, Madrid, Spain

Бектон Дикинсон С.А., Камино де Вальдеолива,
с/н, 28750 Сан Агустин дель Гуадаликс, Мадрид, Испания

Бектон Дікінсон С.А., Каміно де Вальдеоліва,
с/н, 28750 Сан Агустін дель Гуадалікс, Мадрид, Іспанія

Уполномоченная организация: ООО «Бектон Дикинсон Восток»,
127018, РФ, Москва, ул. Двинцев, д.12, стр.1

Представництво в Україні: «Фогт Медікал Фертріб ГмБХ»,
Україна, 03039 Київ, вул. Голосіївська, 7

BD, BD Logo and all other trademarks are property of Becton, Dickinson and Company. © 2014 BD
www.bd.com

DGP15801

Rev. 2014-06