

RICHARDSON HEALTHCARE LIMITED

Document Category: FORM- IFU

Code: F048

Version: 1

Title: Richardson Burns Dressing

Author: Erica Callender

Authorised By: Mayur Patel

Authorised On: 16-Jul-2019

Review On: 16-Jul-2021

Authorised:



Date: 16/07/2019

UNCONTROLLED WHEN PRINTED

CONFIDENTIAL - THIS DOCUMENT CONTAINS CONFIDENTIAL AND PROPRIETARY INFORMATION. THIS DOCUMENT MAY NOT BE REPRODUCED, USED, OR DISCLOSED TO ANY THIRD PARTY WITHOUT PRIOR WRITTEN CONSENT.

Burns Dressing 45cm × 45cm 12ply

Product code: 958610

Indications:

- Absorption of bodily fluids

Instructions for use:

1. Open package and unfold swabs using aseptic technique.
2. Ensure burns area is suitably protected with a non-adherent layer before gently placing swabs without applying pressure.
3. Swabs may be folded for difficult areas or for extra absorbency but must not be cut.

Removal:














1. Swabs are removed gently to ensure no adherence to the wound, saline is recommended for removal where adherence has happened.
2. Swabs are removed based on the condition of the underlying wound or saturation.
3. Dispose as a clinical waste according to local protocols.

Precaution

1. Do not use the product, if the package seal is not intact
2. Do not use on infected wounds
3. This is a single use product, do not reuse
4. Product is supplied sterile, do not re-sterilise

UNCONTROLLED WHEN PRINTED

CONFIDENTIAL - THIS DOCUMENT CONTAINS CONFIDENTIAL AND PROPRIETARY INFORMATION. THIS DOCUMENT MAY NOT BE REPRODUCED, USED, OR DISCLOSED TO ANY THIRD PARTY WITHOUT PRIOR WRITTEN CONSENT.

	EN: Use by date		EN: Manufacturer Richardson Healthcare Ltd 310 Centennial Avenue Centennial Park WD6 3TJ
	EN: Sterilised using ethylene oxide		EN: Date of manufacture
	EN: Do not use if package is damaged		EN: Batch code
	EN: Do not re-use		EN: Keep away from sunlight
	EN: Keep dry		EN: CE marking 0120
	EN: Upper limit of temperature		EN: Not made with natural rubber latex
	EN: Attention: Read all warnings and precautions in instruction for use		x

UNCONTROLLED WHEN PRINTED

CONFIDENTIAL - THIS DOCUMENT CONTAINS CONFIDENTIAL AND PROPRIETARY INFORMATION. THIS DOCUMENT MAY NOT BE REPRODUCED, USED, OR DISCLOSED TO ANY THIRD PARTY WITHOUT PRIOR WRITTEN CONSENT.