




Prepare the skin according to the standard hospital protocol for dressing application. Skin prep or hair removal may be required on some patients for better adhesion.




1
Open the Velcro® lid.

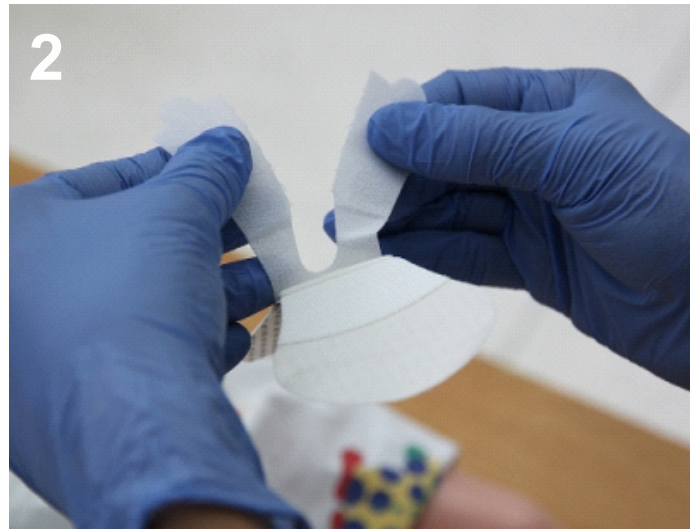

2
Split the lid along the perforations.



3
Place the bifurcation crotch centrally on the Grip-Lok®.

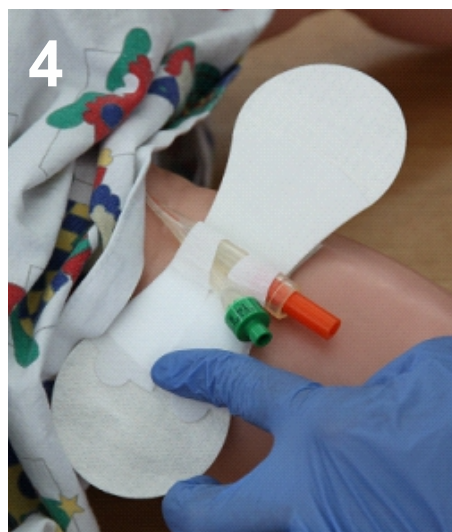

4
Place half the Velcro® lid over the top of the catheter and secure down onto the Velcro®. Then place the remaining half in between the bifurcation crotch and secure down onto the Velcro®.



5
Pull the paper backing from one side of the Grip-Lok®. Then pull the paper backing from the other, to secure the Grip-Lok® in the desired position on the skin.