Symbols Glossary

(Glosář symbolů / Symbole und Glossar / Ordliste over symboler / Glosario de símbolos / Merkinntät Sanasto / Glossaire de symboles / Γλωσσάρι συμβόλων / Szimbólumok jegyzéke / Glossario dei simboli / Verklaring van symbolen / Slowniczek symboli / Glossário de símbolos / Symbolförklaring)

	Use-by date / Použitý / Haltbarkeitsdatum / Holdbarhedsdato / Fecha de caducidad / Viimeinen käyttöpäivä / Date limite d'utilisation / Χρήση έως / Felhasználhatóság ideje / Data di scadenza / Vervaldatum / Data ważności / Data de validade / Använd före		Do not re-use / Nepoužívejte opakovaně / Nicht wiederverwenden / Genbrug aldrig / No reutilizar / Ei saa käyttää uudelleen / Ne pas réutiliser / Μην επαναχρησιμοποιείτε / Egszser használatos / Non riutilizzare / Niet hergebruiken / Nie używać ponownie / Não reutilizar / Får ej återanvändas
	Do not re-sterilize / Opakovaně nesterilizujte / Nicht erneut sterilisieren / Må ikke gensteriliseres / No volver a esterilizar / Älä steriloit uudelleen / Ne pas restériliser / Μην επαναλαμβάνετε την αποστείρωση / Ne sterilizálja újra! / Non risterrilizzare / Niet opnieuw steriliseren / Nie sterylizować ponownie / Não Voltar a Esterilizar / Steriliserä inte på nytt		Authorized European Representative / Autorizovaný zástupce pro Evropu / Bevollmächtigter Vertreter in der EU / Autoriseret europæisk repræsentant / Representante europeo autorizado / Valtuutettu edustaja Euroopassa / Représentant européen agréé / Εξουσιοδοτημένος αντιπρόσωπος στην Ευρώπη / Hivatalos európai képviselő / Rappresentante europeo autorizzato / Geautoriseerde Europese vertegenwoordiger / Autoryzowany Przedstawiciel w Europie / Representante europeu autorizado / Auktoriserad europeisk representant
	Sterilized using Ethylene Oxide (EO) / Sterilizováno etylenoxidem / Mit Ethylenoxid sterilisiert / Steriliseret med ethylenoxid / Esterilizado con / óxido de etileno / Steriloitu etyleenioksidillä / Stérilisation à l'oxyde d'éthylène / Αποστειρωμένο με οξείδιο / του αιθυλενίου / Etilén-oxidál sterilizálva / Sterilizzato con ossido di etilene / Gesteriliseerd met behulp / van ethylenoxide / Sterylizowano tenkiem etylenu / Esterilizado por Óxido de Etileno / Steriliserad med etylenoxid		Consult instructions for use / Viz návod k použití / Gebrauchsanweisung beachten / Læs brugsanvisningen / Consulte las instrucciones de uso / Katso käyttöohjeita / Consultez le mode d'emploi / Συμβουλευτείτε τις οδηγίες χρήσης / Olvassa el a használati utasítást / Consultare le istruzioni per l'uso / Raadplaaq de gebruiksinstructies / Zapoznać się z instrukcją użytkowania / Ver as Instruções de Utilização / Se bruksanvisningen
	Not made with natural rubber latex / Není zhotoven s použitím latexu z přírodního kaučuku / Ohne Naturkautschuklatex / Ikke fremstillet med naturlig gummlatex / No fabricado con látex de goma natural / Ei valmistettu luonnonkumilatekista / Ne contient pas de latex de caoutchouc naturel / Δεν κατασκευάζεται από λατέξ από φυσικό καουτσούκ / Nem tartalmaz természetes gummlatexet / Non prodotto con lattice di gomma natural / Niet gemaakt met natuurlijk rubber latex / Produkt nie jest wykonany z naturalnej gumy kauczukowej / Não fabricado com látex de borracha natural / Ej tillverkad av naturlig gummlatex		Do not use if package is damaged / V případě poškozeného obalu nepoužívejte / Nicht verwenden wenn Packung beschädigt ist / Må ikke bruges, hvis pakken er beskadiget / No utilizar si el embalaje está dañado / Ei saa käyttää, jos pakkaus on vahingoittunut / Ne pas utiliser si l'emballage est endommagé / Ne μην χρησιμοποιείτε εάν η συσκευασία έχει υποστεί ζημιά / Ne használja, ha a csomagolás sérült / Non utilizzare se la confezione è danneggiata / Niet gebruiken bij beschadigde verpakking / Nie używać, jeśli opakowanie jest uszkodzone / Não utilize se a embalagem estiver danificada / Använd ej produkten om förpackningen är skadad
	Manufacturer / Výrobce / Hersteller / Producent / Fabricante / Valmistaja / Fabricant / Κατασκευαστής / Gyártó / Produttore / Fabrikant / Producent / Fabricante / Tillverkare		Batch Code / Kód šarže / Batch-Code / Partikode / Código de lote / Erän tunnus / Code de lot / Κωδικός παρτίδας / Gyártási tételszám / Codice di lotto / Batchcode / Kod partii / Código do lote / Partikod
	Caution / Upozornění / Achtung / Forsigtig / Precaución / Huomautus / Attention / Προσοχή / Vigyázat / Attensione / Let op / Uwaga / Cuidado / Warning		Catalogue number / Katalogové číslo / Bestellnummer / Katalognummer / Número de catálogo / Tuotenumero / Référence (catalogue) / Αριθμός καταλόγου / Katalógusszám / Numero di catalogo / Catalogusnummer / Numer katalogowy / Número de Catálogo / Katalognummer
	Quantity / Počet / Menge / Antal / Cantidad / Määrä / Quantité / Ποσότητα / Mennyiség / Quantità / Aantal / Jakość / Quantidade / Kvantitet		Content / Obsah / Inhalt / Indhold / Contenido / Sisältö / Contenu / Περιεχόμενο / Tartalom / Contenuto / Inhoud / Zawartość / Conteúdo / Innehåll

MICROTEK™ Probe Covers


Instructions for Use

Table of Contents

EN Probe Covers Instructions for Use	2
CZ Návod k použití návrků na sondu	3
DE Gebrauchsanweisung für Sondentücher	4
DK Brugsanvisning til probebeskyttelsestønder	5
ES Instrucciones de uso de fundas para sonda	6
FI Sondin suojusten käyttöohjeet	7
FR Mode d'emploi des couvre-sondes	8
GR Οδηγίες χρήσης καλύμματος ανιχνευτή	9
HU Szondaborítók – használati utasítás	10
IT Istruzioni per l'uso dei coprisonda	11
NL Gebruiksaanwijzing sondehoes	12
PL Instrukcja użytkowania osłon na sondę	13
PT Instruções de utilização das protecções para sonda	14
SE Bruksanvisning för sondskydd	15

Operational Instructions

(Pokyny k použití / Bedienungsanweisungen / Brugsvejledning / Instrucciones de funcionamiento / Käyttöohjeet / Consignes d'utilisation / Consignes d'utilisation / Οδηγίες λειτουργίας / Használati utasítások / Istruzioni di funzionamento / Bedieningsinstructies / Instrukcje dotyczące użytkowania / Instruções de Funcionamento / Användningsinstruktioner)

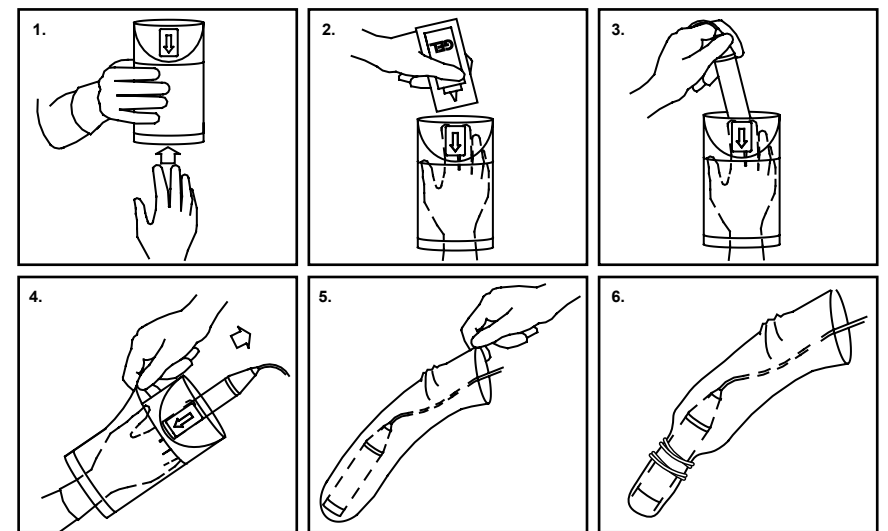Microtek Medical, Inc.
602 Lehmberg Road
Columbus, MS 39702 U.S.A.
www.microtekmed.com

EC REP Microtek Medical B.V.
Hekkehorst 24
7207 BN Zutphen, The Netherlands

EN Probe Covers

Instructions for Use

Rx ONLY CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician.

Intended Use / Indications for Use

A device intended to be used as a physical barrier for protection against the effects of environmental exposure (e.g., body fluids, gels) and/or to maintain the required hygienic level of various diagnostic or surgical procedures which utilize a transducer (probe).

This is a single use device. [Not to be reprocessed.]

The devices are further indicated to be used in or on:

- Intact or compromised skin
- Natural body orifices (e.g., vagina, rectum, esophagus)
- Intra-operatively (e.g., on internal organs)

Contraindications

There are no known contraindications for these devices.

Warnings

- Not intended for contact with the Central Nervous System
- Contents supplied STERILE using an ethylene oxide (EO) process. Do not use if sterile barrier package is damaged. If damage is found, call your Ecolab-Microtek representative.

Precautions

Secure distal tip to probe with rubber bands or ties to prevent gel from migrating away from tip.

Product / Device Description

Probe Covers are composed of natural or synthetic materials and come in a variety of sizes and shapes to allow these devices to properly fit equipment with diverse shapes and uses.

Probe Covers are used in a variety of clinical and surgical settings by trained professionals, such as:

- **General Purpose** – For use in biopsy, trauma, emergency room procedures, patients in isolation, catheter and drainage sites
- **Intraoperative** – For use in biopsy, abdominal surgery, cardiac surgery, vascular surgery, and special procedures
- **Endocavity** – For use in transvaginal, transrectal and prostate imaging

The devices cover a transducer / probe and, in some instances, the cables connected to the transducers / probes. These coverings offer a physical barrier between the transducer and the patient.

How Supplied

- Do not use if package is opened or damaged, store in a cool, dry, dark place.
- This product may be supplied with ultrasound gel, rubber bands and/or ties.

Method of Sterilization

Sterilized using Ethylene Oxide (EO)

Reuse Warning Statement

For single patient use only. Do not reuse, reprocess or resterilize. Reuse, reprocessing or resterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Reuse, reprocessing or resterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.

Handling and Storage

- After use, dispose of product and packaging in accordance with hospital, administrative and/or local government policy.

Warranty Statement

The manufacturer warrants that this product will meet the manufacturer's standard written specifications then in effect and will be fit for the uses and purposes described in manufacturer's literature. As the sole remedy for breach of this warranty, Manufacturer will, in its reasonable discretion, either replace any product that did not meet this warranty or refund the purchase price paid for those products. Handling, storage, cleaning and sterilization of this product as well as other factors relating to the patient, diagnosis, treatment, surgical procedures and other matters beyond manufacturer's control directly affect the product and the results obtained from its use. This warranty does not apply to (i) damage resulting from misuse, neglect, accident or improper use of any product by any person or entity other than manufacturer or (ii) any product altered by any person or entity other than manufacturer. ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING ANY WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, ARE DISCLAIMED. FURTHER, MANUFACTURER ASSUMES NO LIABILITY WITH RESPECT TO PRODUCTS REUSED, REPROCESSED OR RSTERILIZED AND MAKES NO WARRANTIES, EXPRESS OR IMPLIED, INCLUDING BUT NOT LIMITED TO MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, WITH RESPECT TO SUCH PRODUCTS. MANUFACTURER ALSO DISCLAIMS LIABILITY FOR ALL CONSEQUENTIAL, INCIDENTAL, AND/OR SPECIAL DAMAGES CAUSED BY OR RELATED TO THE PRODUCTS.

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