Certus – Camera Cover

GB

Please read the following information thoroughly before using this device.

Description:

The disposable camera sleeve can be used during any endoscopic procedure to create a sterile field around head and lead.

The disposable Camera Sleeve is a sterile, single-use, disposable device and is manufactured using polyethylene.

Contraindications:

This device is not intended for use when endoscopic techniques are contraindicated.

Preparation:

- 1) Using an aseptic technique, open the packaging and drop the device into the sterile field.
- 2) Insert the camera head through sleeve and remove the paper from the front. Insert the sterile telescope.
- 3) Remove the blue sealing tape and wrap the tape around the sleeve, covering the sheath of the telescope. Use additional blue tape as necessary.
- 4) The scrub nurse should hold the telescope with one hand and push the ring and attached sleeve down the cable. The circulating nurse can then pull the ring further down the cable. The circulating staff must not come into contact with any part of the drape which will remain in the sterile field.
- 5) The second sealing tape may be used to stabilize the sleeve as necessary.
- 6) Upon completion of the procedure this product must be disposed of in accordance with local regulations.

Precautions

The Disposable Camera Sleeve is designed for single use only, and is not to be re-used or sterilized.

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