

# BD Eclipse™ Needle with SmartSlip™ Technology

Aguja Eclipse™ con tecnología SmartSlip™  
 Agulha Eclipse™ com tecnologia SmartSlip™  
 Aiguille Eclipse™ avec la technologie SmartSlip™  
 Eclipse™-Kanüle mit SmartSlip™-Technologie  
 Ago Eclipse™ con tecnologia SmartSlip™  
 Eclipse™ naald met SmartSlip™ technologie  
 Eclipse™ nål med SmartSlip™-teknik  
 Eclipse™-kanyle med SmartSlip™ teknologi  
 Eclipse™-neula, jossa SmartSlip™-teknologia  
 Τεχνολογία βελόνας Eclipse™ με SmartSlip™  
 Eclipse™-kanyle med SmartSlip™-teknologi  
 Igla Eclipse™ z zastosowaniem techniki SmartSlip™  
 Igla Eclipse™ s tehnologijo SmartSlip™  
 Ihla Eclipse™ s tehnoloogiaga SmartSlip™  
 SmartSlip™ tehnoloogiaga Eclipse™ nõel  
 Eclipse™ tú SmartSlip™ tehnológiával  
 Eclipse™ adata su SmartSlip™ tecnologia  
 Jehla Eclipse™ s tehnologijí SmartSlip™  
 Eclipse™ adata ar SmartSlip™ tehologiju  
 SmartSlip™ Teknologijí Eclipse™ Igne  
 Игла Eclipse™ с технологией SmartSlip™  
 Igla Eclipse™ s tehnologijom SmartSlip™  
 Ac Eclipse™ cu tehnologie SmartSlip™  
 Eclipse™ Игла със SmartSlip™ Технологии  
 Голка Eclipse™, яка виготовлена за технологією SmartSlip™



INSTRUCTIONS FOR USE  
 INSTRUCCIONES DE USO  
 INSTRUÇÕES DE UTILIZAÇÃO  
 MODE D'EMPLOI  
 GEBRAUCHSANWEISUNG  
 ISTRUZIONI PER L'USO  
 GEBRUIKSINSTRUCTIES  
 BRUKSANVISNING  
 BRUGSANVISNING  
 KÄYTTÖOHJEET  
 ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ  
 BRUKSANVISNING  
 INSTRUKCJA UŻYCI

NAVODILA ZA UPORABO  
 POKYNY PRE POUŽITIE  
 KASUTUSJUHEND  
 HASZNÁLATI UTASÍTÁS  
 NAUDOJIMO INSTRUKCIJA  
 NÁVOD K POUŽITÍ  
 LIETOŠANAS NORĀDĪJUMI  
 KULLANMA TALIMATI  
 ИНСТРУКЦИИ ПО ПРИМЕНЕНИЮ  
 UPUTA ZA UPORABU  
 INSTRUCȚIUNI PENTRU UTILIZARE  
 INSTRUKCII ZA UPOTREBA  
 INSTRUKCIJA PO ZASTOSYWANIU

## English



1) Push firmly when attaching the needle to the syringe. Pull back on the safety cover. Grasp the syringe with one hand and with the other hand pull the needle shield straight off.



2) Activate safety mechanism immediately after injection. Center your thumb or forefinger on the textured finger pad and push the safety cover forward over the needle until you hear or feel it lock. Visually confirm that the needle is covered. If unable to activate, discard immediately in an approved sharps container.




3) Use one handed technique and activate away from self and others. For greatest safety, ONLY use the wide textured finger pad area to activate the safety cover.

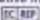
Activation of the protective mechanism may cause minimal splatter of any fluid that is remaining on the needle after injection.

Discard after single use in an approved sharps container in accordance with applicable regulations and institutional policy.

Non-pyrogenic. Do not use if individual packaging is damaged.

This product does **NOT** contain natural rubber latex. Do Not Reuse 

Manufacturer 

Authorized Representative in the European Community 

### CAUTION

Where local and/or institutional procedures permit/require transportation of the filled syringe, use a passive recapping technique to cover the needle before transporting to the point of administration.

Re-use may lead to infection or other illness/injury.

USA only: OSHA standards require that such recapping must be accomplished using a one handed technique. **DO NOT** hold the needle shield during the recapping process.



USA only: Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.

To help avoid HIV (AIDS), HBV (Hepatitis) and other infectious diseases due to accidental needlesticks, activate the protective mechanism immediately after use.

Do not autoclave BD Eclipse™ Needle before use.

STERILE EO



 BD, Franklin Lakes, NJ 07417 USA  
 БД, Фрэнклин Лэйкс, НДж 07417 США  
 BD, Laagstraat 57, B9140, Temse, Belgium  
 БД, Лаахстраат 57, 59140, Темсе, Бельгия

Представительство компании «Бектон Дикинсон Б.В.» (Нидерланды),  
 123317 Россия, Москва, Краснопресненская наб., 18  
 Becton Dickinson İthalat İhracat Ltd. Şti., Büyükdere Cad. Üçyol Mevkii  
 Noramin İş Merkezi No: 55 K: 1/105, 34398 Maslak İstanbul, Türkiye

Made in Spain [www.bd.com](http://www.bd.com)

BD, BD Logo and all other trademarks are property of Becton, Dickinson and Company. ©2011 BD

DGP74P203 8365117 Rev. 2011-11

