



blunt fill and blunt filter needles

instructions for use



Medicina Ltd.

Unit 1-4 Rivington View Business Park,
Station Road, Blackrod, Bolton, BL6 5BN, UK
Telephone: 01204 695050

CE
1 6 3 9



DEVICE DESCRIPTION:

Medicina blunt fill BN01 and Medicina blunt filter needles BN02

INDICATIONS AND INTENDED USE:

Drawing up medicines from ampoules.

BN01 for glass vials with rubber stopper and plastic ampoules

BN02 for use with glass ampoules.

INTENDED USER:

Medical professionals.

WARNINGS/ PRECAUTIONS:

A recent Adverse Event was reported whereby 'coring' could occur when using the Blunt Fill Needle with a specific Pharmaceutical drug. This specific drug was the antibiotic Piperacillin/Tazobactam. Testing has shown this not to be attributable to the Blunt Fill Needle; however please remain extra vigilant if using the Blunt Fill Needle with this drug.


CONTENT OF THE PACKAGE:

BN01 Blunt fill needle

BN02 Blunt filter needle

DIRECTIONS:

1. Secure Hub of needle to syringe by twisting clockwise. Syringe is ready to access single dose vials or multiple dose vials or ampoules.
2. Remember to puncture rubber stopper at a 90 degree angle to reduce the risk of catching the inside wall of the vial.
3. Fill syringe from single dose vial or multiple dose vial. Medicina Blunt fill filtered needle to be used with glass ampoules only. The Medicina blunt fill filtered needle contains a one way filter and should be removed after the syringe is filled.
4. Detach needle from syringe by twisting anti clockwise. Discard needle in an approved sharps container.
5. The syringe is now ready to be used with:
 - Skin injections
 - Luer activated valves
 - IV bags
 - IV systems

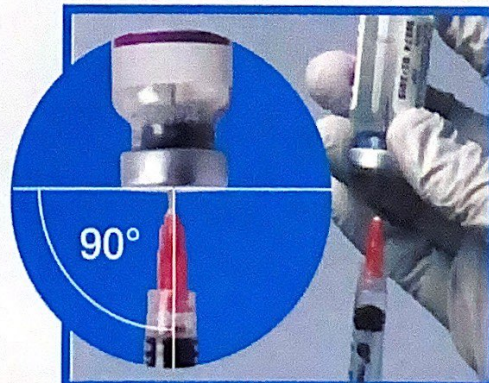

STORAGE CONDITIONS

Keep away from sunlight

Keep dry