SampLok® Adapter Cap 3

INDICATIONS FOR USE (EN)

SampLok Adapter Cap 3 (SAC3) is intended for use to facilitate collection of blood samples into culture bottles or vacuum tubes. It is a single use device. There are no known contraindications for the SAC3, though care should be taken per the warnings and precautions included in this IFU.

WARNINGS AND PRECAUTIONS

SampLok Adapter Cap 3 is a single-use device. Do not attempt to remove the needle after use.

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INSTRUCCIONES FOR USE

- Attach a collection needle to the SampLok Adapter Cap 3. Screw the blood collection needle into the opening at the top of the cap until it is tight (A).
- Skin preparation: Prepare and disinfect venipuncture site following your facility procedure.
- Culture bottle preparation: Remove the culture bottle protective cap (B). Disinfect culture bottle septum as per culture bottle

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manufacturer instructions (C).

- Perform venipuncture following your facility procedure.
- Collect blood sample into culture bottle.
a. Place SampLok Adapter Cap 3 over culture bottle and press down to start flow into the culture bottle (D).
- Lift SampLok Adapter Cap 3 off the culture bottle to stop flow, following your facility procedure for sample volume guidelines (E). Repeat steps 5a. and 5b. for additional culture bottles.
- Collect blood sample into vacuum tubes (as required). If using Cap Insert, place into SampLok Adapter Cap 3 (F).
a. Insert vacuum tube (G).

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Samplok® es una marca registrada de ITL Corporation, Canberra, Australia.