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Issued April 2020

	Keep away from sunlight		Caution		Temperature limit
	Keep dry		Does not contain natural rubber latex		Do not use if package is damaged
	Fragile, handle with care		Manufacturer		Do not reuse
	This way up		Date of manufacture		Do not re-sterilise
REF	Catalogue number		Use-by date		Sterilised using ethylene oxide
LOT	Batch code	CE 1639	Conformance to European Medical Device Directives		Consult instructions for use
QTY	Quantity	EC REP	Authorised Representative in the European Community		

Lapro Surge Limited

5b Fishers Industrial Estate, Wiggan Road, Watford, Herts WD18 0FN, UK
Tel: +44 (0)1923 220 711 Fax: +44 (0)20 8950 8659
www.laprosurge.com

EC REP Advena Limited

Tower Business Centre, 2nd Flr., Tower Street, Swatar, BKR 4013 Malta

How Supplied

Contents are sterile for single use only.

Do not use if the package is opened or damaged.

Do not re-sterilise.

Contraindications

This device is not intended for use when endoscopic techniques are contraindicated.

Cautions/Warnings

1. This device is designed for single use only, and is not to be reused or re-sterilised. Please do not re-use or re-process to avoid irreversible injury such as cross-infection or injury caused by device malfunction. Do not Use if package is opened or damaged.
2. Endoscopic procedures should be performed only by physicians with adequate training and knowledge of these procedures. In addition, medical literature should be consulted regarding techniques, hazards, contraindications, and complications prior to the performance of these procedures.
3. Before endoscopic instruments and accessories from different manufacturers are utilised together in a procedure, verify compatibility and ensure that electrical isolation of ground is not compromised.
4. Store this device in a dry, clean and safe place. Keep the device away from flammable and combustible chemicals or materials.
5. Please follow universal guidelines upon medical waste disposal to avoid any potential cross-infection, environmental contamination or injury caused by sharp waste.

Intended Use

1. The Insufflation Tubing Set is for use to insufflate the abdominal cavity in order to create working space for endoscopic procedure, and it contains a filter for filtration of the insufflation gas.
2. The insufflation tubing is available with either a luer lock or a 15mm ISO connector for compatibility with a range of insufflators.

Instructions for Use

1. Connect FT004 or Luer lock FT001 adjacent to filter housing to a suitable Insufflator.
2. Connect luer lock end of tubing to Laparoscopic cannula gas tap.
3. Set Insufflator flow rate according to Insufflator manufacturer's instructions.

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