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EN-GB ENGLISH

How Supplied

Contents are sterile for single patient use only.
Do not use if the package is opened or damaged.
Do not re-sterilise.

Contraindications

This device is not intended for use when endoscopic techniques are contraindicated.

Cautions/Warnings

1. This device is designed for single use only, and is not to be reused or re-sterilised. Do not Use if package is opened or damaged.
2. Endoscopic procedures should be performed only by physicians with adequate training and knowledge of these procedures. In addition, medical literature should be consulted regarding techniques, hazards, contraindications, and complications prior to the performance of these procedures.
3. Before endoscopic instruments and accessories from different manufacturers are utilised together in a procedure, verify compatibility and ensure that electrical isolation of ground is not compromised.
4. A thorough understanding of techniques and principles involved in electrosurgical procedures is necessary to avoid burn or shock hazard to patient and/or operator.
5. Do not embed the tip of the electrode in the tissue for more than 10 seconds.
6. If electrocautery is required, this instrument should be used with certified monopolar electrosurgical generators. Keep the HF power output as low as possible to achieve the desired end effect.
7. Refer to instructions for use provided by the manufacturer of electrosurgical generator to ensure that all safety precautions are followed. Using a metal trocar sheath in conjunction with a plastic site stability device creates a potential hazard when using electrosurgical instruments.
8. Certain applications of the HF device can cause sparks. Follow the safety instructions in the HF device's instructions for use very carefully.
9. Prior to switching on the HF device, make sure that the product's working tip is not touching any accessory or fluid that conducts electricity. Keep the working tip in the user's visual field whenever HF power is activated.
10. Do not put down the instrument on the patient or in the immediate vicinity of the patient.

Intended Use

1. The Suction Irrigation set is a sterile multifunctional instrument which achieve up to 3 functions in one, Suction, Irrigation and Coagulation. Single packed accessories are available as following: Monopolar L hook, Spatula, and Electrode probes.
2. The Laprosurge Suction Irrigation Set is available with an array of probe designs to facilitate lavage during laparoscopic surgery. This device has applications in laparoscopic gynaecologic, general, thoracic and urology procedures to provide suction and irrigation functions to help flush blood and tissue debris from the operative site during laparoscopy to aid visualization.

Instructions for Use

Maximum Diathermy Setting: Cutting 50W, Coagulation 70W.

1. Close the white tubing clamp before connecting to the irrigation pump. Remove the spike cap and insert the spike to the desired irrigation pump.
2. Connect suction adapter (end of clear tubing) to patient port of suction bottle. Please follow instructions for use provided by the manufacturer of suction machine.
3. Insert the probe through the desired 5mm or 10mm trocar sheath.
4. Press the white trumpet valve on the handlepiece for suction, the blue trumpet valve for irrigation. Please also refer to instructions provided by manufacturer of suction machine and irrigation pump for safe operation.
5. When electrocautery is required, ensure that the outer sleeve of the monopolar probe is fully retracted to expose the Spatula or L Hook tip. To expose the tip, pull back the black ring on the monopolar probe.
6. Note: Do not use suction or irrigation during electrosurgical procedure. Verify that electrode tips are not in contact with suction or irrigation fluid path. Do not embed the tip of electrode in the tissue for more than 10 seconds.
7. Upon completion of the endoscopic procedure, dispose this device in accordance with local regulations.

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