



# IV FLUID INFUSION SET

with 15µm filter, air vent, 180cm tubing. 3-way connector, 120cm extension tubing, rotating luer connector



## INSTRUCTIONS FOR USE :

1. Remove set from package, close clamp.
  2. Prepare I.V. Container.
  3. Remove spike protector, squeeze drip chamber and insert spike through outlet hole in container.
  4. Suspend container. Set fluid by squeezing and releasing drip chamber.  
DO NOT FILL CHAMBER COMPLETELY.
  5. Open clamp to expel air bubbles. Close clamp, remove protector from adapter end and attach to I.V. Cannula.
  6. Set flow rate by adjusting clamp.
- 'For use with I.V. bags or solution bottles'

•NON-TOXIC •NON-PYROGENIC



### CAUTION :

- Do not use if package is damaged.
  - For single use only.
  - Discard after single use.
  - Do not use for blood transfusion.
  - Do not use if protective caps are loose or missing
- "GRAVITY FEED ONLY"

**ECRep** Obelis S.A  
 Boulevard Général Wahis 53  
 1030 Brussels, BELGIUM  
 T + 32 (0) 2732 5954  
 E mail@obelis.net

**Distributed in ROI by:**  
 Fannin Ltd.,  
 Fannin House,  
 South County Business Park,  
 Leopardstown, Dublin 18  
 T + 353 (0) 1290 7000  
 E info@fannin.eu



### 3mm ID TUBING

**REF** : 1029DFX

Priming Volume :

**LOT** :

:

:

Code No. :

**CE**  
0123

**STERILE EO**



**DEHP FREE**  
**LATEX FREE**



**Fannin**  
 caring for life since 1829

**Distributed in UK by:**  
 Fannin (UK) Ltd.,  
 Westminster Industrial Estate,  
 Repton Road, Measham,  
 Swadlincote, Derbyshire,  
 DE12 7DT  
 T + 44(0)1453 874130  
 E salesll@fannin.eu

**POLY MEDICURE LTD.**

Plot No.104-105, Sector-59, HSIIDC Indl. Area,  
 Faridabad-121004, (Haryana) INDIA.

Rev.07-20/11