

## RICHARDSON HEALTHCARE LIMITED

Document Category: FORM- IFU

Code: F048

Version: 1

Title: ClearFilm IV

Author: Erica Callender

Authorised By: Mayur Patel

Authorised On: 16-Jul-2019

Review On: 16-Jul-2021

Authorised:



Date: 16/07/2019

**UNCONTROLLED WHEN PRINTED**

**CONFIDENTIAL** - THIS DOCUMENT CONTAINS CONFIDENTIAL AND PROPRIETARY INFORMATION. THIS DOCUMENT MAY NOT BE REPRODUCED, USED, OR DISCLOSED TO ANY THIRD PARTY WITHOUT PRIOR WRITTEN CONSENT.

---

**Clearfilm I.V 7cm x 8.5cm**  
**Product Code 816061**

**\*\*This product is non-sterile and needs to be sterilised prior to use\*\***

**Indications:**

- Central Line Catheter dressing
- PICC Line Catheter dressing
- Midline Catheter dressing

**Application:**

1. Open package and remove sterile dressing.
2. Position dressing over catheter site at the desired location.
3. Ensure transparent film window is over insertion site and apply dressing.
4. Cross over the soft cloth section under lumens.
5. Slowly remove the frame while firmly smoothing down the dressing edges to maximise fixation.
6. The additional two fixation strips can be applied for further support.
7. Enter the dressing change details onto the record label and fix onto dressing, ensuring the window is not obscured.

**Removal:**

1. It is essential that you support the skin and catheter by using finger pressure while removing the dressing.
2. Gently remove fixation strips.
3. Separate the soft cloth under the lumens.
4. Secure the catheter with one hand, and using the other hand grasp the edge of the dressing and slowly release the dressing while remaining parallel to the skin. Continue this method around all edges.
5. Support the catheter at all times to avoid displacement.

**Precautions:**

1. Site of application must be dry before application.
2. Haemostasis of the catheter site or wound should be achieved before applying the dressing.
3. Do not stretch the dressing during application as tension can cause skin trauma.

	<b>EN: Use by date</b>		<b>EN: Manufacturer</b> Richardson Healthcare Ltd 310 Centennial Avenue Centennial Park WD6 3TJ
	EN: Not made with natural rubber latex		<b>EN: Date of manufacture</b>
	<b>EN: Upper limit of temperature</b>		<b>EN: Batch code</b>
	<b>EN: Do not re-use</b>		<b>EN: CE marking</b>
	<b>EN: Keep dry</b>		EN: Keep away from sunlight

**UNCONTROLLED WHEN PRINTED**

**CONFIDENTIAL** - THIS DOCUMENT CONTAINS CONFIDENTIAL AND PROPRIETARY INFORMATION. THIS DOCUMENT MAY NOT BE REPRODUCED, USED, OR DISCLOSED TO ANY THIRD PARTY WITHOUT PRIOR WRITTEN CONSENT.