

RICHARDSON HEALTHCARE LIMITED

Document Category: FORM- IFU

Code: F048

Version: 2

Title: Softpore

Author: Erika Callender

Authorised By: Mayur Patel

Authorised On: 04-Apr-2019

Review On: 04-Apr-2021

Authorised:



Date:04/04/2019

UNCONTROLLED WHEN PRINTED

CONFIDENTIAL - THIS DOCUMENT CONTAINS CONFIDENTIAL AND PROPRIETARY INFORMATION. THIS DOCUMENT MAY NOT BE REPRODUCED, USED, OR DISCLOSED TO ANY THIRD PARTY WITHOUT PRIOR WRITTEN CONSENT.

Softpore Dressing

Indications:

- Surgical wounds
- Minor burns
- Cuts and abrasions

Instructions for use:

1. Measure wound site to ensure correct size of dressing.
2. Open the package and remove the dressing.
3. Remove one section of backing paper, exposing adhesive surface.
4. Fixate dressing ensuring wound is centred to the pad
5. Remove rest of the backing paper, smooth border firmly to obtain adhesion.
6. Do not stretch the dressing when applying.

Frequency of change:

SoftPore should be changed according to the clinical condition of the wound or when saturated. The dressing may be left in the situ for up to 5 days.

Removal:















1. Hold edge of dressing, slowly peel in direction of hair growth.
2. Dispose as a clinical waste according to local protocols.

Precaution:

1. To prevent skin shearing do not apply under tension.
2. Dressing should not be applied to patients who are known to be sensitive to acrylic adhesives or to individuals who have very fragile or easily damaged skin
3. Single use only. Do not use.
4. Product is supplied sterile, do not re-sterilise.

UNCONTROLLED WHEN PRINTED

CONFIDENTIAL - THIS DOCUMENT CONTAINS CONFIDENTIAL AND PROPRIETARY INFORMATION. THIS DOCUMENT MAY NOT BE REPRODUCED, USED, OR DISCLOSED TO ANY THIRD PARTY WITHOUT PRIOR WRITTEN CONSENT.

	EN: Use by date		EN: Manufacturer Richardson Healthcare Ltd 310 Centennial Avenue Centennial Park WD6 3TJ
	EN: Sterilised using ethylene oxide		EN: Date of manufacture
	EN: Do not use if package is damaged		EN: Batch code
	EN: Do not re-use		EN: Universal recycling
	EN: Keep dry		EN: Keep away from sunlight
	EN: Upper limit of temperature		EN: CE marking 0120
	EN: Attention: Read all warnings and precautions in instruction for use		EN: Not made with natural rubber latex

UNCONTROLLED WHEN PRINTED

CONFIDENTIAL - THIS DOCUMENT CONTAINS CONFIDENTIAL AND PROPRIETARY INFORMATION. THIS DOCUMENT MAY NOT BE REPRODUCED, USED, OR DISCLOSED TO ANY THIRD PARTY WITHOUT PRIOR WRITTEN CONSENT.