

Devon™

Needle Counter

Compteur d'aiguilles
Nadelzähler
Conta aghi
Contador de agujas
Nålräknare
Naaldenteller
Contador de agulhas
Neulojenlaskin
Nåletæller
Μετρητής βελόνων
Ročítač jehel
Tűszámláló
Кейс для игл
Ројемник на иглы
İğne Sayacı
Nåleteller
Ročítadlo ihiel
Casetă pentru ace
Кутия-номератор
за игли

STERILE R



Single use

IFU No. KS77221436



COVIDIEN, COVIDIEN with logo and Covidien logo are U.S. and/or internationally registered trademarks of Covidien AG.

Other brands are trademarks of a Covidien company.

© 2011 Covidien.

Covidien llc, 15 Hampshire Street, Mansfield, MA 02048 USA.
www.covidien.com

REV 05/2010

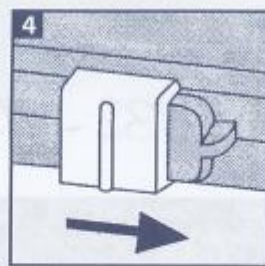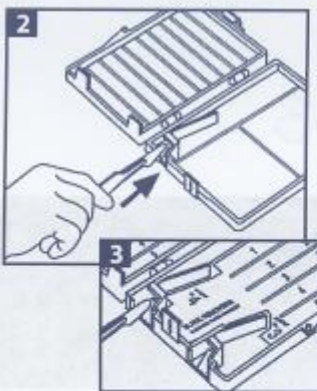

en
Directions for Use

Precautions:

Do not open package until immediately prior to use.

Blade Removal Instructions

1. Hold scalpel handle in cutting position, with sharp edge of blade down.

2. Insert scalpel into the blade remover opening until it can enter no further. (A slight "click" may be heard just as it approaches full insertion.)

3. Pull straight back on the scalpel handle.

If Equipped with BoxLocks closure:

4. Slide both yellow "BoxLocks" forward (toward front edge of box) until they click into place.

For European BladeGuard II Use Only:

Select the correct blade remover for the proper scalpel.