

◆ PRODUCT INFORMATION◆

NEEDLE COUNTER (SHARPS BOX)

◆ Description:

The NEEDLE COUNTER is designed for disposal of surgical blades and needles after procedures. It is available in various sizes with several foams and magnetic pads for use in operation rooms.






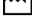









◆ Preparation:

- 1) Remove the sleeve containing the NEEDLE COUNTER from the peel pouch package. Open the box and fix to the procedure area using the adhesion tape.
- 2) Put the used sharp blades and needles into the glue area. After procedure, close it completely.
- 3) To dispose of the NEEDLE COUNTER:
The used NEEDLE COUNTER must be discarded in compliance with government regulations for infectious waste disposal.

◆ Cautions/Warnings:

- Completely inspect package prior to use. Do not use if the sterile packaging is opened, damaged or wet. Sterility might be compromised.
- Do not use product after expiration date!
- Do not re-use! If re-used, sterility will be compromised.
- Do not re-sterilize!
- "To be disposed according regulations of respective country." or "Dispose used product following local state or federal guidelines."
- "For use by or on the order by a physician, or persons licensed by state law."

Explanation of symbols:

	Caution		Sterilized using Ethylene Oxide
	Do not use if package is damaged		Batch code
	Keep away from sunlight		Date of manufacture YYYY-MM
	Catalogue number		Manufacturer
	Do not re-sterilize		Use by
	Consult instructions for use		Do not reuse
	Keep dry		Authorised representative in the European community.
	Notified body: DNV GL Presafe AS, with CE2460		

Distributed By:



Fannin UK LTD
Unit 100, Westminster Industrial Estate,
Repton Road, Measham, Swadlincote, DE12 7DT
Tel:01453 874130

Manufactured By:



Advanced Medical Design Co., Ltd.
4F~5F, No. 29, Wuquan 5th Rd., Wugu Dist.,
24888 New Taipei City, Taiwan



Authorized Address:



Lotus NL B.V.
Koningin Julianaplein 10, 1e Verd, 2595AA, The Hague,
Netherlands