

# ◆ PRODUCT INFORMATION◆

## EASI GRIP (LIGHT HANDLE COVER)

### ◆ Description:

The EASI GRIP is designed to provide a sterile field for the surgeon to grip the Operating Room light during a procedure. It is designed for single use and not for re-sterilization.
















### ◆ Preparation:

- 1) Remove the sleeve containing the EASI GRIP from the peel pouch package. Tear the film liner off.
- 2) Then assemble onto Operating Room light handle directly.
- 3) To dispose of the EASI GRIP:  
The used EASI GRIP must be discarded in compliance with government regulations for infectious waste disposal.

### ◆ Cautions/Warnings:

- Completely inspect package prior to use. Do not use if the sterile packaging is opened, damaged or wet. Sterility might be compromised.
- Do not use product after expiration date!
- Do not re-use! If re-used, sterility will be compromised.
- Do not re-sterilize!
- "To be disposed according regulations of respective country." or "Dispose used product following local state or federal guidelines."
- "For use by or on the order by a physician, or persons licensed by state law."

### Explanation of symbols:

	Caution		Sterilized using Ethylene Oxide
	Do not use if package is damaged		Batch code
	Keep away from sunlight		Date of manufacture YYYY-MM
	Catalogue number		Manufacturer
	Do not re-sterilize		Use by
	Consult instructions for use		Do not reuse
	Keep dry		Authorised representative in the European community.
	Notified body: DNV GL Presafe AS, with CE2460		

Distributed By:



**Fannin UK LTD**  
Unit 100, Westminster Industrial Estate,  
Repton Road, Measham, Swadlincote, DE12 7DT  
Tel:01453 874130

Manufactured By:



Advanced Medical Design Co., Ltd.  
4F~5F, No. 29, Wuquan 5th Rd., Wugu Dist.,  
24888 New Taipei City, Taiwan



Authorized Address:



Lotus NL B.V.  
Koningin Julianaplein 10, 1e Verd, 2595AA, The Hague,  
Netherlands