



## Silicone Vessel ID Loops INSTRUCTIONS FOR USE SINGLE USE ONLY

### INTENDED USE

The “Fortune” Silicone Vessel ID Loops are used in complicated surgical procedures by offering retraction, occlusion, and identification of arteries, nerves, veins, tendons or ureters.

### Patient target group

Patients who require the retraction, occlusion, and identification of arteries, nerves, veins, tendons, or ureters during surgical procedures.

### Intended user

General surgeon.

### FEATURES

1. 100% medical grade silicone for superior biocompatibility
2. Soft and smooth material for better comfort
3. Silicone radiopaque
4. Non-adherent and non-absorbent to tissue
5. Non-toxic and latex-free
6. Sterile
7. Only for use during surgery

### SPECIFICATION

Cat No.	Size	Description
9005-0503R	Width: 0.5mm Thick: 0.3mm Length: 450mm	Red color
9005-0503B		Blue color
9005-0503Y		Yellow color
9005-0503W		White color
9005-1309R	Width : 1.3 mm Thick : 0.9 mm Length : 450 mm	Red color
9005-1309B		Blue color
9005-1309Y		Yellow color
9005-1309W		White color
9005-2510R	Width : 2.5 mm Thick : 1.0 mm Length : 450 mm	Red color
9005-2510B		Blue color
9005-2510Y		Yellow color
9005-2510W		White color
9005-2510A		Red, blue, yellow, white color
9005-5013R	Width: 5.0mm Thick: 1.3mm Length: 450mm	Red color
9005-5013B		Blue color
9005-5013Y		Yellow color
9005-5013W		White color

### DISPOSAL

When disposing this device, it should be done according to your country’s regulations for similar types of medical waste.

### STORAGE

The product should be kept in dry and cool place.

### STERILIZATION:

1. The Silicone Vessel ID Loops is a single-use item and supplied sterile.
2. Sterilization is effected by E.O. Gas, as indicated on the label.
3. Please see the label on the sterilized peel-pouch for the expiration date.
4. Do **not** re-sterilize or reuse. Reused Silicone Vessel ID Loops may cause inflammation and bacterial infection.
5. Sterile only if the package has not been opened, damaged or broken.

### PRECAUTION:

Operators applying excessive force may result in the tube fractured during use. Be aware to use the device carefully.

### Notice

If any serious incident has occurred concerning this device please report it to Fortune Medical Instrument Corp. and the competent authority.

**Graphical Symbols for Medical Device Labeling**

Symbols	Indication
	Catalogue number
	Date of manufacture
	Use-by date
	Batch code
	Consult instructions for use
	Sterilized using ethylene oxide
	Keep dry
	Keep away from sunlight
	Do not re-use
	Do not re-sterilize
	Do not use if package is damaged
	Avoid sharp tool
	Manufacturer
	Authorized representative in the European Community
	Caution
	Medical device
	Single sterile barrier system with protective packaging inside
	Importer



**fortune**  
medical



FORTUNE MEDICAL INSTRUMENT CORP.

6F., No. 29, Sec. 2, Zhongjheng E. Rd., Danshuei Dist,  
New Taipei City 251, TAIWAN

Manufactured by : No. 256. Changchun 2nd Rd., Jhongli  
Dist, Taoyuan City 320, TAIWAN

TEL: (886) 2 2624 2233      Web: [www.fortunemed.com](http://www.fortunemed.com)

FAX: (886) 2 2624 2266      Email: [info@fortunemed.com](mailto:info@fortunemed.com)



[Emergo Europe](http://www.fortunemed.com)  
Prinsessegracht 20, 2514 AP The Hague, The  
Netherlands

