

K-LITE

K-LITE LONG

Product Information Data-card

PRODUCT DESCRIPTION:

K-Lite is a lightweight knitted fabric consisting of 93% Viscose, 4% Nylon and 3% elastomeric yarn. Due to its knitted structure, the bandage has two-way stretch giving it excellent conformability and ease of use

K-Lite is a **LATEX FREE** product

INDICATIONS:

K-Lite is a **Type 2 Light Support bandage**, designed to be used for sprains and strains and for the prevention of oedema.

K-Lite is a suitable lightweight, conformable and cost effective alternative to the traditional crepe bandage.

K-Lite is not suitable for the application of pressure on its own and should therefore only be used in combination with other bandages in the treatment of leg ulcers caused by venous insufficiency.

K-Lite is one of the components in the K-Four multi-layer compression bandage kit.

METHOD OF USE:

The bandage should be applied in a spiral over the dressing to secure it in place.

For compression bandaging it is applied in a spiral with a 50% overlap.

FREQUENCY OF DRESSING CHANGES:

The bandage may be left in place for up to a week or at the discretion of the clinician.

CONTRA-INDICATIONS:

K-Lite should not be used on anyone with an allergy to any one of the component properties

PRESENTATION:

K-Lite bandages are individually shrink-wrapped.

K-Lite Long 10cm x 5.25m is included in the **K-Four** multi-layer compression kit for ankles >25cm.

SIZES:

K-Lite

5cm x 4.5m (Stretched)*

7cm x 4.5m (Stretched)*

10cm x 4.5m (Stretched)*

15cm x 4.5m (Stretched)*

K Lite Long

10cm x 5.25m (stretched)*

*Available on Drug Tariff

