

English

BARD® Urological Catheter

Warning: Do not use ointments or lubricants having a petrolatum base. They will damage silicone.

Visually inspect the product for any imperfections or surface deterioration prior to use.



- Use luer tip syringe to inflate with stated mL of sterile water.
OR
- For pre-filled products, remove clip and squeeze reservoir to inflate with stated mL of sterile water.

Storage: Store catheters at room temperature away from direct exposure to light, preferably in the original box.

For urological use only.


To deflate catheter balloon: Gently insert a luer slip tip syringe in the catheter valve. Never use more force than is required to make the syringe "stick" in the valve. Allow the pressure within the balloon to force the plunger back and fill the syringe with water. *If you notice slow or no deflation, re-seat the syringe gently.* Use only gentle aspiration to encourage deflation if needed. Vigorous aspiration may collapse the inflation lumen, preventing balloon deflation. If necessary, contact adequately trained professional for assistance, as directed by hospital protocol.

Should balloon rupture occur, care should be taken to assure that all balloon fragments have been removed from the patient.

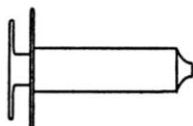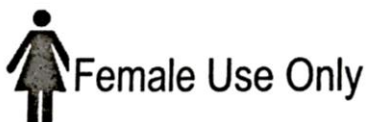

Syringe of Sterile Water for
balloon inflation

Refer to Direct Unit Label for Product Content and Gender Specific Use where applicable.

 Single use only.

 Do not resterilize

This is a single use device. Do not resterilize any portion of this device. Reuse and/or repackaging may create a risk of patient or user infection, compromise the structural integrity and/or essential material and design characteristics of the device, which may lead to device failure, and/or lead to injury, illness or death of the patient.

LOT

Lot Number

REF

Reorder Number



Use by
Date



Caution, consult
accompanying
documents.



Consult
instructions for use.



Units



Do Not Use if Package is Damaged.

STERILE R Sterilized by radiation

OR

STERILE EO

Sterilized by Ethylene
Oxide

Refer to Direct Unit Label for Sterilization Method Utilized

 Bard, Bardex, Bardia, Biocath and Lubricath are trademarks and/or registered trademarks of C. R. Bard, Inc.



Warning: After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable laws and regulations.

BARD



Manufacturer:
C. R. Bard, Inc.
Covington, GA 30014 USA
1 800 526 4455
www.bardmedical.com

EC REP



0086

Bard Limited
Forest House
Crawley, West Sussex
RH11 9BP UK
+44 1293 527 888

PK8704019 PK590 08/2011