



USE INSTRUCTIONS

for PullUp™ Probe Cover for Probe Cover

INTENDED PURPOSE / INTENDED USE

This is a medical device used over imaging probes or instruments to provide a sterile protection, and to minimize cross contamination during diagnostic imaging procedures.

USE INSTRUCTIONS

Prepare probe per system's operational manual.

1. **PullUp™** | Squeeze sides to open cover.
2. **Probe Cover** | Open probe cover.
3. Place gel inside cover and/or on probe scanning surface.
4. Insert probe into cover.
5. Pull cover over probe and cord. Use tape or bands to fasten cover. Use sterile gel between cover and patient.

This product is single-use only.

Do not reuse. Do not resterilize. Reuse, reprocessing or re-sterilization of the single use devices may create a risk of cross contamination, patient infection, and / or malfunction of the device.

After use, dispose of single-use components as infectious waste. Do not use if sterile package is damaged.

STERILE EO



Do not use if sterile package is damaged.



Do not re-use.



Not made with natural rubber latex.



For translations of this instruction, go to www.protekmedical.com.
For more information contact Protek Medical Products, Inc.
1-041-0102 REV. 15, ISSUE DATE: 04-2019
Copyright © 2012 Protek Medical Products, Inc.



Protek Medical Products, Inc., 4125 Westcor Court, Coralville, Iowa 52241, USA

CE 2797