



UK Smith & Nephew, Croxley Park, Building 5, Hatters Lane, Watford, Hertfordshire WD18 8YE. Tel. 08000 157 573

AUSTRALIA Smith & Nephew Pty Ltd, 85 Waterloo Road, North Ryde, NSW 2113. Tel. (02) 9857 3999

AUSTRIA Smith & Nephew GmbH, Concorde Business Park C3, A-2320 Schwechat. Tel: +43 (0) 707 91 02

BELGIUM Smith & Nephew N.V.-S.A., Hector Henneaulaan 366 - 1930 Zaventem. Tel: (02) 702 29 11

CANADA Smith & Nephew Inc., 2280 Argentia Road, Mississauga, ON L5N 6H8. Tel: 1 800 463 7439

CHINA Smith & Nephew Ltd, 15F, One Museum Place Office Building, No 669 Xinzha Road, Jing'an District, Shanghai, China. Tel: +86 (021) 23303000

DENMARK Smith & Nephew A/S, Slotsmarken 14, 2970 Hørsholm. Tel: +45 45 80 61 00

EASTERN EUROPE Smith & Nephew GmbH, Concorde Business Park C3, A-2320 Schwechat. Tel: +43 (0) 707 91 02

FINLAND Smith & Nephew Oy, Ayrntie 12 C, 01510 Vantaa. +358 207 866 300

FRANCE Smith & Nephew SAS, 40/52 boulevard du Parc, F-92200 Neuilly Sur Seine. Tel: +33 (0) 800 111 220

GERMANY Smith & Nephew GmbH, Friesenweg 4/ Haus 21, 22763 Hamburg. Tel. 040 879744-0

GREECE Costas A. Papaellinas (Hellas) SA, 26th km, Paeanias - Markopoulou Ave, PO Box 182, 19400 Koropi. Tel: 662 6201-7

HONG KONG Smith & Nephew Ltd., Unit 813-818, 8/F, Delta House, 3 On Yiu Street, Shatin, N.T. Tel: (852) 2648 7700

INDIA Smith and Nephew HealthCare Pvt. Ltd, B-501-509, Dynasty Business Park, Andheri-Kurla Road, Andheri (E), Mumbai-400059. Tel: 022 4005 5090

IRELAND Smith & Nephew Limited, Carraig Court, George's Avenue, Blackrock, Co. Dublin, Tel: 01-217 0444

ITALY Smith & Nephew S.r.l. - via De Capitani 2A, 20864 Agrate Brianza (MB). Tel: +39 039 60941

JAPAN Smith & Nephew KK, 2-4-1 Shiba Koen, Minato-ku, Tokyo 105-0011 Japan. Tel. 03-5403-8930

KOREA Smith & Nephew Ltd, 13F, ASEM Tower 517, Yeongdong-daero, Gangnam-gu, Seoul, Korea (135-798). Tel: 2 6001 7575

MALAYSIA Smith & Nephew Healthcare Sdn. Bhd. Suite 11.01B & 11.02, Level 11, Menara AmFIRST, No 1 Jalan 19/3, 46300 Petaling Jaya, Selangor. Tel: +603-7958 7103

MALTA Cherubino Ltd, Delf Building, Sliema Road, Gzira. Tel: 21 343270

MEXICO Smith & Nephew, SA de CV, Patricio Sanz No. 442, Colonia del Valle, C.P. 03100, México D.F. Tel. 53 40 22 60

MIDDLE EAST Smith & Nephew FZE, 4th Floor National Bank of Abu Dhabi Building, Al Jazari Building 47, Dubai Healthcare City - District 8, PO Box 9715, Dubai, U.A.E. Tel. 971 4 4299111

NETHERLANDS Smith & Nephew Nederland CV, Bloemlaan 2, 2132 NP Hoofddorp, The Netherlands. Tel: (+31) (0) 20 6543999

NEW ZEALAND Smith & Nephew Limited, Unit A, 36 Hillside Road, Wairau Valley, Auckland 0627. Tel. 9 820 2840

NORWAY Smith & Nephew A/S, PO Box 224, Nye Vakas vei 64, 1379 Nesbru, Tel. 66842020

PAKISTAN BSN Medical (PVT) Ltd, A/69 SITE, Manghopir Road, PO Box 3659, Karachi 75700. Tel. + 92 21 32561176-77

POLAND Smith & Nephew Sp. z o.o. Ul. Osmańska 12, 02-823 Warszawa D +48 22 360 41 20

PORTUGAL Smith & Nephew Lda., Estrada Nacional n° 10, Km 131, Parque Tejo - Bloco C, 2625-445 Forte da Casa. Tel: +351 214 460 650

SINGAPORE Smith & Nephew Pte Ltd., 29 Media Circle, #06-05 Alice @ Mediapolis Singapore 138565 Tel: (+65) 6270 0552

SOUTH AFRICA Smith & Nephew (Pty) Limited 30 The Boulevard, West End Office Park, Westville, 3629. Tel: 031 2428111

SPAIN Smith & Nephew SAU, Fructuós Gelabert, 2-4, 08970 Sant Joan Despí, Barcelona. Tel: 93 373 73 01

SWEDEN Smith & Nephew AB, Box 143, Kråkelorpsgatan 20, 43122 Molndal. Tel: 031-746 58 00

SWITZERLAND Smith & Nephew Schweiz AG, Advanced Wound Management, Obemeuhofstrasse 10d, 6340 Baar. Tel: +41 41 766 22 44

TAIWAN (R.O.C.) Smith & Nephew (Overseas) Ltd, Taiwan Branch, 9F-2, No. 50, Sec. 1, Xinseng South Road Taipei City, Tel: 886 2 3322 5020

THAILAND Smith & Nephew Ltd, 16th Floor Building A, The 9th Towers Grand Rama 9, 33/4 Rama 9 Road, Huaykwang Sub-District, Huaykwang District, Bangkok 10310. Tel: 66(0)2-116-5581

TURKEY Smith ve Nephew Medikal Clazlar Tic. LTD. Şti. Bahçeköy Mah. Orkide Sokak No: 8/E 34473 Sarıyer/İSTANBUL. Tel: +90 212 226 22 55

UNITED STATES Smith & Nephew, Inc, 5600 Clearfork Main Street, Suite 600, Fort Worth, TX 76109. Customer Care Center: 1-817-900-4000

VENEZUELA Ace Sales Corp, 8085 NW 68th Street, Miami, Florida 33166 Tel: 305.835.0310

GR Na μην χρησιμοποιείται εάν η συσκευασία είναι ανοικτή ή κατεστραμμένη

DK Må ikke anvendes hvis pakningen er åben eller beskadiget

NO Må ikke brukes hvis emballasjen er åpnet eller skadet

FI Ei saa käyttää, jos pakkaus on avattu tai vahingoittunut

SE Använd inte om förpackningen är öppen eller skadad

PT Não utilizar se a embalagem estiver aberta ou danificada

TR Paket açılmış ya da hasar görmüş ise kullanmayınız



(EN) Do not use if pack is open or damaged

(FR) Ne pas utiliser si l'emballage est ouvert ou endommagé

(DE) Nicht verwenden wenn Verpackung bereits geöffnet oder beschädigt ist

(ES) No utilizar si el envase está abierto o deteriorado

(IT) Non utilizzare se la confezione è aperta o danneggiata

(NL) Niet gebruiken indien verpakking geopend of beschadigd is

EN

smith&nephew

IV3000°

Moisture responsive catheter dressing
With securing strips and documentation label

Description

IV3000° is a moisture responsive transparent film dressing, specifically designed to meet the needs of catheter fixation and IV site protection.

Mode of action

IV3000 is made from a unique REACTIC™ film, which is highly breathable film with a high-permeability to water vapour as compared to other film technologies. The film protects the IV insertion site from external environmental contaminants as a physical barrier to liquids, particulate matter and microbial contaminants. IV3000 is spread with a low allergy adhesive. Sterile medical-grade securing strips and a documentation label are also provided with the dressing.

Instructions for use

Clean and thoroughly dry the application area in accordance with normal procedures

1. Remove the securing strips from the printed carrier paper (1) and apply as required, ensuring that the strips do not cover the insertion site. The strips can be used on the catheter wings or hub for improved security, to secure tubing or to stabilise catheter lumens. Remove carrier (1) from the dressing.
2. Remove the protector paper (2) in the direction indicated in the diagram.
3. Place the dressing on the catheter hub and adhere the dressing to the patient's skin ensuring minimal creasing of the product. Remove the remaining protector paper (3).
4. Remove the printed transparent carrier (4) by lifting the non-adhesive tab at the end of the dressing and pulling diagonally until the carrier is removed.
5. Mould the film around the catheter to complete dressing application.
6. In accordance with local clinical protocols, information may be recorded with a ballpoint pen on the documentation label before removing from the carrier (1) and being placed in

the patient's notes, on the skin or on the dressing (without obscuring the view of the catheter insertion site)

7. If using more than one dressing, do not allow the dressings to overlap
8. Remove dressing by loosening the film and stretching the dressing parallel to the patient's skin while stabilising the patient's skin and catheter with the other hand. Alternatively, use an alcohol wipe to break the adhesive seal. If the skin is particularly fragile then warm soap and water may be used to break the adhesive bond and to ease the dressing off.

Indications

The above-mentioned product is suitable for securing non-ported catheters.

Contraindications

IV3000 should be used on IV sites only. If a film dressing is required for open wounds such as pressure sores, burns or donor sites, OPSITE® FLEXIGRID® should be used. The securing strips and documentation label are not indicated as a replacement for sutures or other primary wound closure methods.

Precautions

Use on prepared dry skin only. Do not stack dressings, or allow dressings to overlap. Periodically monitor the dressing and catheter site to confirm secure attachment and continued proper infusion, especially after bathing, showering, or if the dressing and catheter site becomes wet. If the dressing detaches, evaluate to ensure proper catheter placement then apply a new dressing. As with all adhesive products, apply and remove carefully from sensitive or fragile skin. If reddening or sensitisation occurs, discontinue use. IV3000 is a single use only product. If used on more than one patient, cross-contamination or infection may result. Opening the dressing pack compromises the sterile barrier therefore any unused dressing should not be retained for application at a later date.

Product Availability

Code	Size	Indication	Availability	Ported
4007	6cm x 7cm / 2 ³ / ₁₆ in. x 2 ³ / ₁₆ in.	Peripheral catheters	Carton of 100	No
4008	10cm x 12cm / 4in. x 4 ³ / ₁₆ in.	Central catheters	Carton of 50	No
66004005	10cm x 12cm / 4in. x 4 ³ / ₁₆ in.	Central catheters	Carton of 10	No

Other IV3000 Products Available

Code	Size	Indication	Availability	Ported
66004011	5cm x 6cm / 2in. x 2 ³ / ₁₆ in.	Peripheral catheters	Carton of 100	Yes
4006	7cm x 9cm / 2 ³ / ₁₆ in. x 3 ¹ / ₁₆ in.	Peripheral catheters	Carton of 100	Yes
66004009	9cm x 12cm / 3 ¹ / ₁₆ in. x 4 ³ / ₁₆ in.	Peripheral and central catheters	Carton of 50	Yes
66800512	11cm x 14cm / 4 ⁵ / ₁₆ in. x 5 ¹ / ₁₆ in.	Peripherally inserted central catheters	Carton of 25	Yes

Smith & Nephew Medical Ltd, 101 Hessele Rd, Hull HU3 2BN England.
Trade Marks of Smith & Nephew ©Smith & Nephew Date of issue 03/2020
www.smith-nephew.com

