

BD Spinal Needle

- SPINAL NEEDLE Quincke Type Point
- AGUJA ESPINAL, punta tipo Quincke
- Agulha espinal de ponta Quincke
- AIGUILLE SPINALE Biseau de Quincke
- SPINALKANÜLE mit Quinckeschliff
- AGO SPINALE:Punta tipo Quincke
- Spinale naald met Quincke punt
- SPINALNÄL med Quincke slipad spets
- SPINALKANYLE med Quincke slib
- SPINAALINEULA, Quincke tyurppinen kärki
- СПИНАЛЬНАЯ ИГЛА с заточкой типа Quincke
- ΒΕΛΟΝΑ ΟΣΦΥΟΝΩΤΙΑΙΑΣ ΠΑΡΑΚΕΝΤΗΣΗΣ με Άκρο τύπου Quincke
- Igła podrażczynówkowa z końcówką typu Quincke point
- Spinál tű Quincke típusú hegygel
- SPINÁLNÍ JEHLA s Quincke hrotem
- Spinálna ihla s hrotem typu Quincke

- SPINALNA IGLA s konico tipa Quincke
- SPINAALNÖEL Quincke-tüüpi otsikuga
- SPINALINÉS ADATOS Quincke tipo galiukas
- SPINÄLÄ ADATA ar Quincke tipa smaili
- SPINALNÄL med spiss av Quincke-type
- SPINAL İĞNE Quincke Tipi Uç
- ابرة شوكية من نوع كوينكه نقطة
- СПИНАЛНА ИГЛА със скосен връх тип Quincke
- Spinalna igla s Quincke vrhom
- Квинке типті спиналды інне
- Ac spinal cu vârful tip Quincke
- Spinalna igla sa vrhom „Quincke Point”
- Спинална голка з заточкою типу Квінке
- Игла стерильная одноразовая в комплекте с зондом для спинно-мозговой и люмбальной пункции и анестезии BD Spinal Needle (Quincke)



en

OBSERVE UNIVERSAL PRECAUTIONS ON ALL PATIENTS

REPORT NEEDLE STICKS IMMEDIATELY AND FOLLOW ESTABLISHED PROTOCOL.

- Percutaneous puncture with a contaminated needle may lead to serious illness such as hepatitis, HIV (AIDS), or other infectious diseases.

- Resheathing needles is hazardous.

NON-PYROGENIC - STERILE:

Unless package has been opened or damaged.

DO NOT RESTERILIZE.

As with any spinal procedure, to help avoid needle breakage, **DO NOT ATTEMPT TO STRAIGHTEN A BENT NEEDLE.**

Care should be taken to avoid needle damage. Repeated repositioning may increase the risk of needle breakage. Observe needle carefully during procedures.

Remove spinal needle and introducer needle together. Replace with a new needle.

ENSURE SYRINGE IS SEATED FIRMLY IN NEEDLE HUB.

To be used by properly qualified and trained health professionals.

CAUTION: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

Re-use may lead to infection or other illness/injury.



Do not use if package is damaged.

No utilizar si el envase está dañado.

Não utilizar se a embalagem estiver danificada.

Ne pas utiliser si l'emballage est endommagé.

Bei beschädigter Verpackung nicht verwenden.

Non utilizzare se la confezione è danneggiata.

Niet gebruiken indien de verpakking beschadigd is.

Får ej användas om förpackningen är skadad.

Må ikke anvendes, hvis pakningen er skadet.

Ei saa käyttää, jos pakkaus on vahingoittunut.

Не использовать, если упаковка повреждена.

Μην χρησιμοποιείτε το προϊόν αν η συσκευασία είναι κατεστραμμένη.

Nie stosować, jeżeli opakowanie zostało uszkodzone.

Ne használja fel amennyiben a csomag sérült.

Nepoužívajte, ak je obal poškodený.

Nepoužívejte, je-li obal poškozen.

Ne uporabljajte, če je ovojnina poškodovana.

Mitte kasutada, kui pakend on kahjustatud.

Negalima naudoti, jei pakuotė pažeista.

Neizmantot, ja iepakojums ir bojāts.

Må ikke brukes hvis pakken er skadet.

Ambalaj Hasarlıysa Kullanmayın.

لا تستخدم إذا كانت العبوة تالفة

Да не се използва, ако опаковката е нарушена.

Ne koristite ako je ambalaža oštećena.

Қорабы бұзылған жағдайда бұйымды қолданбаңыз.

Nu utilizați dacă ambalajul este deteriorat.

Ne koristiti ako je pakovanje oštećeno.

Не використовувати, якщо упаковка пошкоджена.



STERILE EO

CE 0318



UATR.116



Becton Dickinson S.A., Camino de Valdeoliva, s/n, 28750 San Agustín del Guadalix, Madrid, Spain

Бектон Дикинсон С.А., Камино де Вальдеолива, с/н, 28750 Сан Агустин дель Гуадаликс, Мадрид, Испания

Бектон Дікінсон С.А., Каміно де Вальдеолива, с/н, 28750 Сан Агустін дель Гуадалікс, Мадрид, Іспанія



BD Switzerland Sàrl, Terre Bonne Park A4, Route de Crassier 17, 1262 Eysins, Switzerland



For Switzerland: Imported by: Becton Dickinson AG, Binningerstrasse 94, 4123 Allschwil, Switzerland

Уполномоченная организация: ООО «Бектон Дикинсон Восток», 127018, РФ, Москва, ул. Двинцев, д.12, стр.1

Уповноважений представник в Україні: ТОВ "Кратія Медтехніка", вул. Багговутівська 17-21, 6-й поверх, 04107, Київ, Україна

Иглы стерильные одноразовые для спинно-мозговой и люмбальной пункции и анестезии в комплектах и интродьюсеры