

**SOLO® DENTAL NEEDLES** SOLO NEEDLES: Sterile stainless steel/silicized dental needles with bevel marking

**INDICATIONS**

The needles are indicated for routine local dental anesthetic injections. The needles are intended for maxillofacial injections.

**CONTRAINDICATION**

Hypersensitivity to any component.

**INSTRUCTIONS FOR USE**

- Break the sterile guarantee seal just before use by turning the two parts of the protective sleeve in opposite directions.
- Screw the pre-threaded needle onto the pre-threaded hub of the syringe.
- Remove the plastic cap just before the injection is administered. It can be used again to remove the needle from the syringe to avoid risk of needle stick injury.
- Discard the used needle into a puncture-resistant container in accordance with all applicable with medical waste regulations.

**WARNINGS**

- Never reuse needles by hand. Use only accepted dental recapping procedures. Recapping needles by hand or recapping bent or broken needles can cause accidental needle stick injury.
- Do not bend, break, or otherwise stress needles as serious injuries to you and/or your patient can occur.
- Do not insert needle to hub during injections, as needles can break and become lodged in patient's tissue, potentially causing serious permanent injury.
- Do not use short needles (<30 mm) when the expected depth in soft tissue approaches the length of the needle.
- Avoid extreme pressure and excessive needle movement during injection as this may cause needles to break, resulting in serious injury to you and/or your patient.
- Be especially vigilant when positioning a needle in a patient who appears to be apprehensive.

Solo®	Ø (mm) x L (mm)	Ø (Gø)	Type of Injection
Extra Short	0.3 x 12	30	Intraigamentary
Short	0.3 x 23	30	Periapical
Long	0.4 x 35	27	Nerve Block

**PRECAUTIONS FOR USE:**

- Store in original packaging until ready to use.
- Read the instructions for use supplied with your injection device.
- If seal is broken, the needle **IS NOT STERILE and MUST NOT BE USED.**
- For single use only: this device is for use in a single patient. Any reuse exposes other patients and the user to critical risks, including infection and trauma since the needle is no longer sterile and is more likely to break.
- If multiple injections are required, replace the needle for each cartridge in order to minimize the risk of needle point damage.
- 1 patient / 1 cartridge / 1 needle
- The choice of the needle depends on the use by the practitioner.

**STORAGE**

Do not store above 30°C and protect from moisture.

**PRESENTATION**

100 two-point disposable non-sterilizable dental needles.

(CB) :12/1995

Rev. November 2019