LIGACLIP® EXTRA

Ligating Clips

Clips de Ligature

Ligaturclips

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Clips de ligadura

Ligeerclips

Ligerende clips

Ligeerausklipsit

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Ligeringsclips

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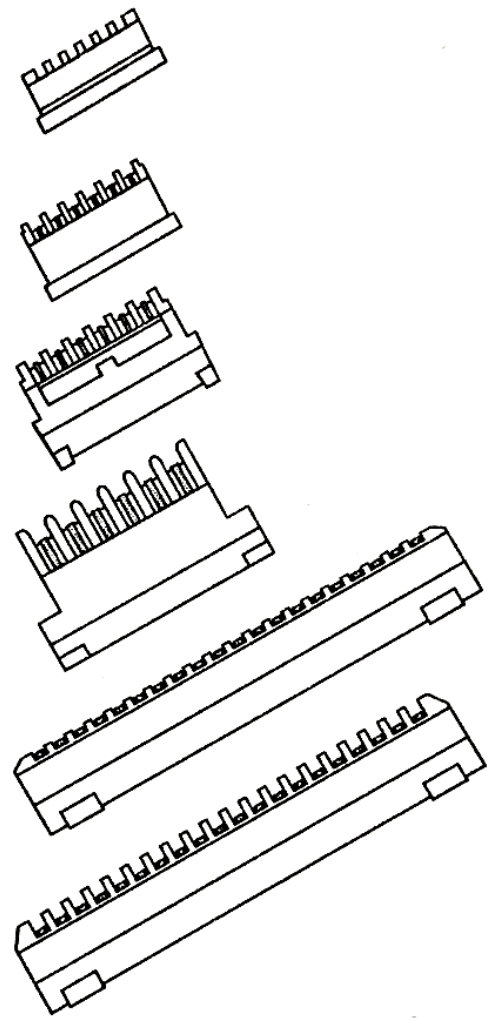
Podvázovacie svorky

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Please read all information carefully.

Failure to properly follow the instructions may lead to serious surgical consequences, such as failure to ligate.

Important: This package insert is designed to provide instructions for use of the LIGACLIP® EXTRA Ligating Clips. It is not a reference to ligation techniques.

LIGACLIP is a trademark of Ethicon Endo-Surgery.

ETHICON ENDO-SURGERY, LLC

a Johnson & Johnson company

Instructions, Instructions, Gebrauchsanweisung, Istruzioni, Instruções, Instrucciones,
Gebruiksaanwijzing, Brugsvejledning, Ohje, Οδηγίες, Bruksanvisning, Instrukcja,
Utasítások, Návod k použití, Návod, 使用说明

REF
LT100, LT200, LT300, LT400, LT102, LT202

Ethicon Endo-Surgery (Europe) GmbH
Hummelsbueteler Steindamm 71
22851 Norderstedt
GERMANY

Ethicon Endo-Surgery, Inc.
4545 Creek Road
Cincinnati, OH 45242-2839 USA
1-877-ETHICON

ETHICON ENDO-SURGERY, LLC
a *Johnson & Johnson* company


ETHICON ENDO-SURGERY, LLC ©EES, LLC 2013
475 Calle C
Guaynabo, PR 00969 USA


Rev. 2013-09

P40347P07

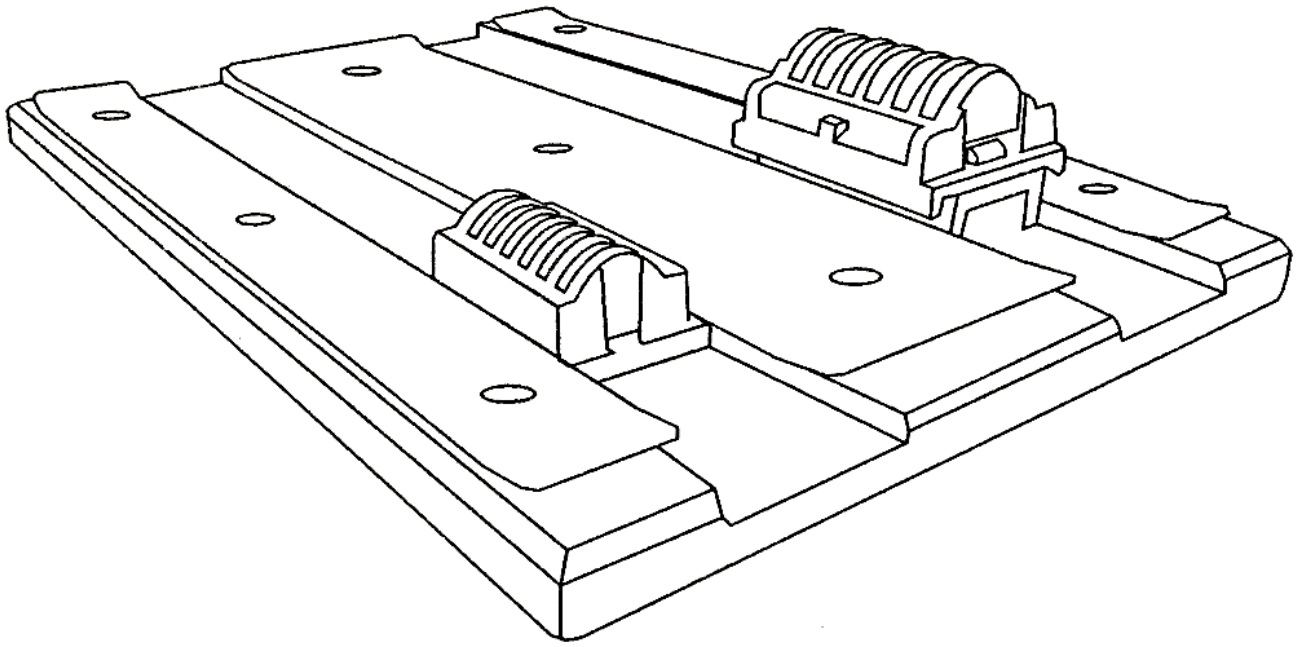1

2

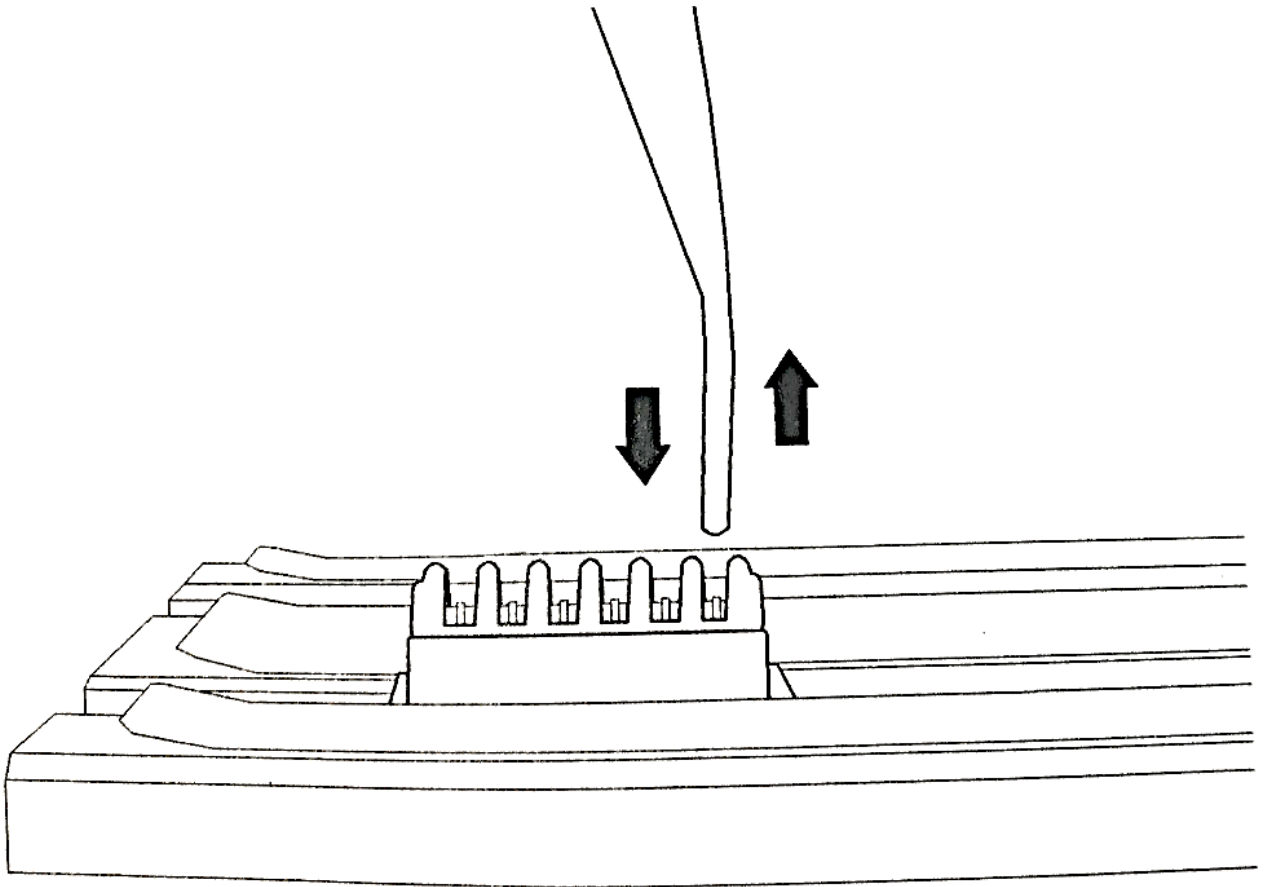Indications

LIGACLIP EXTRA Ligating Clips have application for use on tubular structures or vessels wherever metal ligating clips are indicated. The tissue being ligated should be consistent with the size of the clip.

Contraindications

- DO NOT use the instrument for contraceptive tubal occlusion.
- DO NOT use the instrument on tissue structures or vessels upon which metal ligating clips would not normally be used.

Device Description

LIGACLIP EXTRA Ligating Clips are sterile, single patient use clips produced from titanium. The clips are designed for the ligation of tubular structures where a nonabsorbable ligating device is indicated.

The ligating clips are positioned around a tubular structure and closed by applying pressure to the handles of the applier.

MR Conditional

Non-clinical testing has demonstrated the implantable clips made of titanium in these devices are MR Conditional. A patient with the implanted clips can be scanned safely immediately after placement of the clips, under the following conditions:

- Static magnetic field of 3.0 Tesla or less
- Highest spatial magnetic gradient field of 6.5 Tesla/m
- Maximum MR System reported, whole body averaged specific absorption rate (SAR) of 1.2 W/kg for 20 minutes of scanning (per pulse sequence).

MRI Related Heating

In non-clinical testing, a clip produced a temperature rise of less than 2°C using the following conditions:

- At 3-Tesla (Magnetom Trio Siemens Medical Solutions MR scanner, software version Numaris/4 syngo MRA30), a maximum MR system-reported whole body averaged SAR of 1.2 W/kg
- 20 minutes of continuous MR scanning (per pulse sequence) using transmit/receive RF body coil.

Artifact Information

MR image quality may be compromised if the area of interest is in the same area or relatively close to the position of the clips. Therefore, optimization of MR imaging parameters to compensate for the presence of the clips may be necessary.

The worst case signal void size for a clip was:

Pulse Sequence	SE	SE	GRE	GRE
Plane Orientation	Parallel	Perpendicular	Parallel	Perpendicular
Signal Void Size (mm ²)	199	336	378	348

Instructions for Use

Verify compatibility of all instruments and accessories prior to using the device (refer to **Warnings and Precautions**).

- 1 Using sterile technique, remove the device from the package. To avoid damage, do not flip the device into the sterile field.
- 2 Select the appropriate size LIGACLIP EXTRA Ligating Clip cartridge and corresponding clip applier.
- 3 With the cartridge slots facing away from the base, insert the cartridge into one of the channel openings of the LC800 stainless steel cartridge base. Ensure that the cartridge is past the channel opening and securely held in place (Illustration 1).

- 4 Grasp the applier in the box lock area using pencil grip technique. If using an endoscopic applier, grasp the applier by the handle and trigger.
- 5 Insert the jaws of the instrument into the individual cartridge slot, making sure the tips of the applier are perpendicular to the surface of the cartridge (Illustration 2). Insert the applier until it stops. Do not force the applier. It should enter and withdraw from the cartridge smoothly.
- 6 Keeping the applier jaws perpendicular to the surface of the cartridge, retract the applier from the cartridge (Illustration 2). The clip will be securely held in the applier jaws. It is not necessary to maintain ring or trigger tension to hold the clip in the applier jaws.
- 7 Position the clip around the tubular structure to be ligated. Apply sufficient force to fully close the applier to assure that the clip is satisfactorily placed and secured.

Warnings and Precautions

- Minimally invasive procedures should be performed only by persons having adequate training and familiarity with minimally invasive techniques. Consult medical literature relative to techniques, complications, and hazards prior to performance of any minimally invasive procedure.
- Minimally invasive instruments may vary in diameter from manufacturer to manufacturer. When minimally invasive instruments and accessories from different manufacturers are employed together in a procedure, verify compatibility prior to initiation of procedures.
- A thorough understanding of the principles and techniques involved in laser, electro-surgical and ultrasonic procedures is essential to avoid shock and burn hazards to both patient and medical personnel and damage to the device or other medical instruments. Ensure that electrical insulation or grounding is not compromised. Do not immerse electro-surgical instruments in liquid.
- Ensure that the clip is the correct size for the vessel or tubular structure being ligated.
- Upon each clip application, fully squeeze the handles of the instrument until they come to a stop. Failure to fully squeeze the handles can result in improper formation of clips, and therefore, unsatisfactory ligation of tissue.
- Ensure that each clip has been securely and completely positioned around the tissue being ligated.
- Do not close the jaws of the instrument over other surgical instruments.
- Appliers in disrepair should not be used. The use of such appliers could result in inadequate function of the clip.
- The condition of the LIGACLIP ligating applier, the closure force exerted by the surgeon, the thickness of the tissue to be ligated, and the nature of the clip itself are all critical to the performance of any ligating clip.
- As with any ligating method, inspect each ligation site following placement of the clip to insure that the clip has been satisfactorily placed.
- Dispose of all opened products whether used or unused.
- This device is packaged and sterilized for single use only. Multiple patient use may compromise the device integrity or create a risk of contamination that, in turn, may result in patient injury or illness.

How Supplied

The LIGACLIP EXTRA Ligating Clips are supplied sterile for single patient use. Discard after use.

MR (Magnetic Resonance) Conditional
 Compatible dans certains environnements de résonance magnétique
 Bedingt MR-(Magnetresonanz)-sicher
 Condizionale rispetto alla risonanza magnetica
 Uso condicionado no ambiente de RM (Ressonância magnética)
 Condicionado para RMN (resonancia magnética nuclear)
 MR-conditioneel (MR = magnetische resonantie)
 MR-betinget (magnetisk resonans)
 MR (magneettiresonanssi) -ehdollinen
 Ασφαλές για MR (απεικόνιση μαγνητικού συντονισμού), υπό όρους
 MR (magnetisk resonans)-säker under vissa förutsättningar
 Warunkowo bezpieczny w obszarze oddziaływania rezonansu magnetycznego
 MR (mágneses rezonancia-) vizsgálatra feltételesen alkalmas
 Podmínůnú bezpečné pro MR (MR Conditional)
 Podmienečne vhodné pre použitie magnetickej rezonancie
 MR 磁共振条件

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