

## Suction Instruments Sterile, Single-Use

### Pool Suction Instruments:

1. This device has two parts: instrument shaft and removable sheath.
2. This suction device is indicated for use only by persons trained in suctioning procedures.
3. This suction device is indicated to evacuate pooled blood during deep abdominal surgeries and cesarean sections.
4. This suction device has three positions, providing suctioning capabilities as follows:
  - 4.1 sump (sheath locked in first notch distal from patient)
  - 4.2 fine (sheath locked in second notch distal from patient)
  - 4.3 pinpoint (sheath removed)

### Cautions:

The ability to effectively clean and re-sterilize this single use device has not been established and subsequent re-use may adversely affect the performance, safety and/or sterility of the device.

### Sigmoid Suction Instruments:

1. This suction device is indicated for use only by persons trained in suctioning procedures.
2. This suction device is indicated to evacuate fecal matter during sigmoidoscopy or colonoscopy procedures, and is indicated to evacuate the femur during hip replacement procedures.
3. The winged cap (on/off), controls suctioning pressure to minimize tissue grab.

### Cautions:

The ability to effectively clean and re-sterilize this single use device has not been established and subsequent re-use may adversely affect the performance, safety and/or sterility of the device.