



# Silicone Tubing



## Product Information and Instructions for Use

**MEDASIL SURGICAL LIMITED**  
Medasil House, Hunslet Road  
Leeds, LS10 1AU (UK)  
Tel: +44 (0)113 243 3491  
E-Mail: [orders@medasil.com](mailto:orders@medasil.com)  
Website: [www.medasil.com](http://www.medasil.com)

European Healthcare & Device  
Solutions (Ireland) Ltd Stratton  
House, Bishops Town Road,  
Cork, Ireland T12 Y91C  
Tel: +353 (86) 228 0846  
[www.europeandevicesolutions.eu](http://www.europeandevicesolutions.eu)



SYMBOLS			
<b>REF</b>	Catalogue Number	<b>STERILE R</b>	Manufactured by
<b>LOT</b>	Lot Number	<b>EC REP</b>	Authorized Representative in the European Community
	Use by date (YYYY-MM)	<b>MD</b>	Do Not Use if Packaging is Open or Damaged
	Latex Free		Keep Dry
	Puncture Free		Keep away from sunlight
	Instructions For Use		Do Not Re-sterilize

\*\* Applicable to the Sterile product label  
\*\*\* Applicable to the Non-Sterile product label

## ENGLISH - INSTRUCTIONS FOR USE

**Intended Use:**  
Medasil Silicone Tubing is intended for routine fluid transfer involving suction/aspiration and drainage or other similar applications considered suitable by a Healthcare Professional (HCP).

**Device Description**  
Tubing is manufactured from silicone elastomer which is biocompatible, latex and phthalate free, offered in a range of internal and external diameters. It displays high tear-resistance, tensile strength, and good memory characteristics, capable of resisting distortion when subjected to compression, flexing or stretching. The non-wetting surface properties resist encrustation to facilitate suction/aspiration and drainage applications in clinical settings and can be used for routine fluid transfer processes in a laboratory setting.

**Non-Sterile Tubing:**  
Coils of 15m (±50ft), translucent, red and opaque colour.

**Sterile Tubing:**  
Terminally sterilised using Gamma Radiation.

**Indications for Use:**

- Removal of fluids through suction/aspiration and drainage.

**Contraindications:**

- The Silicone Tubing must not be used for permanent implantation within the body.

**Intended Patient Population**

The Silicon Tubing is intended for use on the general patient population, applied on adults and children.

**Intended Users**

The Medasil Silicone Tubing is intended to be used by the healthcare professionals (HCP).

**Warnings and Precautions:**

Prior to use, consider the following:

- Carefully select the internal and external diameters of tubing to ensure that it meets the needs of each specific application, whilst minimising adverse effects.
- For devices supplied sterile, the sterility is only guaranteed in a sealed, undamaged, and unopened pouch.
- Must not be used if the patient has known allergic reaction towards silicone.
- Must not be used for delivering medicine.
- Must not be used beyond expiry date on the label.
- Must not be used if the packaging has been compromised.
- Must be stored in the original unopened packaging.
- Must be stored away from moisture and direct sunlight.
- Silicone tubing is a single use device.

Additionally, the HCP must ensure that the lumen of the tubing does not become blocked during use by:

- Accumulation of debris.
- Pressure on thin walled tubes causing collapse.
- Sharp bend or kink in tube.

**Single Use:**

Medasil Silicone Tubing has not been validated for re-use. Anyone re-using a single use device, may be held legally liable for the safe performance of the device.

**Sterility:**

Silicone Tubing is supplied either non-sterile or sterile. The STERILE version has been terminally sterilised using Gamma Radiation. Do not re-sterilise.

Non-sterile devices must be sterilised by the end user through validated methods.

**Disposal of Devices:**

Devices must be disposed of in accordance with the local and national regulations.

Table 1: Range of sizes and colours for non-sterile tubing

REF**	Old REF	Non-Sterile Tubing		Colour
		Length (m)	Inner Diameter (mm)	
260-1001-0617	100	0.64	3.95	Translucent
260-1001-1016	101	1.00	4.20	
260-1001-15104	104	1.50	2.80	
260-1001-1516	102	1.50	4.70	
260-1001-1811	109	1.80	4.00	
260-1001-2016	107	1.98	3.18	
260-1001-2019	109	1.98	3.93	
260-1001-2018	109	2.00	5.50	
260-1001-3216	104	3.20	6.40	
260-1001-4012	105	4.00	6.40	
260-1001-4816	106	4.80	7.90	
260-1001-4820	107	5.00	8.70	
260-1001-5010	102	5.00	11.00	
260-1001-6020	108	6.00	10.00	
260-1001-6416	108	6.40	9.50	
260-1001-6425	109	6.40	11.30	
260-1001-6432	110	6.40	11.30	
260-1001-8024	111	8.00	12.70	
260-1001-9516	112	9.50	14.80	
260-1001-9524	113	9.50	15.90	
260-1001-9532	114	9.50	15.90	
262-1001-4504-0115	218R0	1.50	2.30	Opaque Red Opaque
262-1001-4508-0515	233R0	1.50	2.99	
262-1001-2006-0115	217R0	1.98	3.18	

\*Pouch REF: For Individually Packed Devices

Table 2: Range of sizes and colours for sterile tubing

REF**	Old REF	Sterile Tubing		Colour	
		Length (m)	Inner Diameter (mm)		Outer Diameter (mm)
265-1101-0617-0003	100	30	0.64	3.95	Translucent
265-1101-0617-0012	100	120	0.64	3.95	
265-1101-1016-0003	101	30	1.00	4.20	
265-1101-1016-0012	102	120	1.00	4.20	
265-1101-1516-0003	102	30	1.50	4.70	
265-1101-1516-0012	102	120	1.50	4.70	
265-1101-2018-0003	103	30	2.00	5.50	
265-1101-2018-0012	103	120	2.00	5.50	
265-1101-3216-0003	104	30	3.20	6.40	
265-1101-3216-0012	104	120	3.20	6.40	
265-1101-4012-0003	105	30	4.00	7.90	
265-1101-4012-0012	105	120	4.00	7.90	
265-1101-4816-0003	106	30	4.80	7.90	
265-1101-4816-0006	106	60	4.80	7.90	
265-1101-4816-0009	106	90	4.80	7.90	
265-1101-4816-0012	106	120	4.80	7.90	
265-1101-4816-0018	180	180	4.80	7.90	
265-1101-4816-0024	240	240	4.80	7.90	

Sterile Tubing					
REF #1	Old REF	Length (cm)	Inner Diameter (mm)	Outer Diameter (mm)	Colour
261-1101-4820-0003		30			Opaque
261-1101-4820-0006		60			
261-1101-4820-0009		90			
261-1101-4820-0012	107	120		8.70	
261-1101-4820-0018		180			
261-1101-4821-0024		240			
261-1101-5416-0003		30			
261-1101-5416-0006		60			
261-1101-5416-0009		90			
261-1101-5416-0012	108	120		9.50	
261-1101-5416-0018		180			
261-1101-5416-0024		240			
261-1101-6425-0003		30			
261-1101-6425-0006		60	6.40		
261-1101-6425-0009		90			
261-1101-6425-0012	109	120		11.30	
261-1101-6425-0018		180			
261-1101-6425-0024		240			
261-1101-6432-0003		30			
261-1101-6432-0012	110	120			
261-1101-8024-0003		30			
261-1101-8024-0006		60			
261-1101-8024-0009		90	8.00		
261-1101-8024-0012	111	120		12.70	
261-1101-8024-0018		180			
261-1101-8024-0024		240			
261-1101-9516-0003		30			
261-1101-9516-0009		90			
261-1101-9516-0012	112	120			
261-1101-9516-0018		180			
261-1101-9516-0024		240			
261-1101-9524-0003	113	30	9.50	14.30	
261-1101-9524-0012		120			
261-1101-9524-0018		180			
261-1101-9524-0024	114	30		15.80	
261-1101-9532-0012		120			
261-1101-1504-0103	Z13R0	30			
261-1101-1504-0118		180	1.50	2.30	
261-1101-2006-0103		30			
261-1101-2006-0118	Z17R0	180	1.98	3.18	

\*Pouch REF: For Individually Pouched Devices