

Urine collection bag

Bedside drainage bag

Conveen Basic
Conveen Standard
Simpla Profile
Simpla S

Bag hanger / stand

Conveen
Conveen Basic
Simpla



Instructions for use

CM0000000407 Version 2

The Coloplast logo is a registered trademark of Coloplast A/S. All rights reserved © 2014-10-21

Indication

Urine collection bag

Information

Coloplast accepts no liability for any injury or loss that may arise if this product is used in a manner contrary to Coloplast's current recommendations. According to national guidelines it is not recommended to attach a non-sterile bedside drainage bag directly to an indwelling catheter.

Caution

Do not use if package is damaged.


Warning

Re-use of the single use product may create a potential harm to the user. Reprocessing, washing, disinfection, and/or sterilisation may compromise product characteristics, causing additional risk of physical harm to or infection of the user.

How to use a Coloplast bedside drainage bag with Coloplast bag hanger or stand


Preparation

1. A bedside drainage bag can be attached directly to a urethral or indwelling catheter.

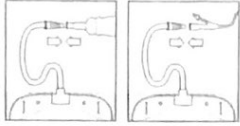


Or it can be connected to a leg bag for extra drainage capacity during the night. For overnight linkage the leg bag must be fitted with a flexible outlet tube that will accommodate the inlet connector of the bedside drainage bag.

Intermittent catheterisation
A bedside drainage bag may also be connected to an intermittent catheter for urinary collection during intermittent catheterisation.

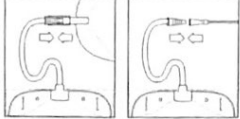

2. If the bag has a sample port, a sample of urine can be taken when necessary.

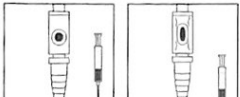

Use a syringe with needle, or a needle free syringe as required.




3. If the tube does not have a pre-attached inlet connector or if the tube itself is not pre-attached to the bag you can shorten the tube to your preferred length. Simply cut the tube with scissors.




When cutting a corrugated tube, leave as much of the straight section as possible in place. This will ensure a secure connection between tube and connector when mounting the connector, or between bag and tube when mounting the tube.

4. Attach the connector to the tube by pushing the thin end of the connector into the tube as far as possible.

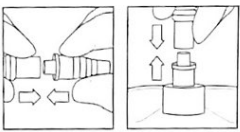

Or attach the tube to the bag by pushing the cut end of the tube firmly into the connection inlet of the bag.

5. Use a Coloplast bag hanger or stand to suspend the bedside drainage bag.

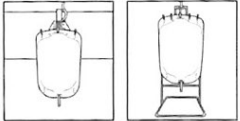


Fit the bag securely on the hanger or stand by inserting the teeth of the hanger or stand in the slits at the top of the bag. Hook the bag hanger onto the bed rail or place the stand on the floor at the bedside.

To ensure optimal drainage, it is important that the bedside drainage bag is positioned lower than yourself when you are lying in bed.


Intermittent catheterisation
Suspend or place the bag as you find most convenient, making sure the bag is positioned lower than your pelvic area whether you prefer to catheterise sitting, standing or lying down.

6. Make sure the outlet tap is closed.

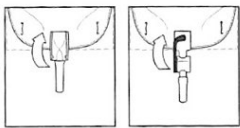

Crimp outlet
Conveen Standard

Lever tap
Simpla Profile

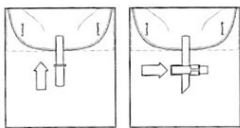


Push-pull outlet
Conveen Basic

Slide action tap
Conveen Basic
Simpla S

1336290

Application

7

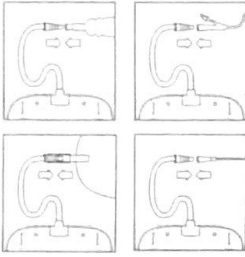

7. If the connector has a protective cap, remove it.

Attach the connector to the urethra or indwelling catheter.

Or attach the connector to the leg bag outlet tube. Open the leg bag outlet tap.


The urinary collecting system is now functional.

Intermittent catheterisation

Attach the connector to the intermittent catheter.

When you begin to catheterise the urine will drain into the bedside drainage bag.

8

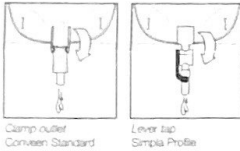


Clamp outlet
Conveen Standard

Lever tap
Simplex Profile

8. We recommend emptying the bag when it is a little over half full to ensure optimal drainage.


Intermittent catheterisation

Empty the bedside drainage bag after each intermittent catheterisation.

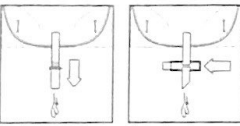

Push-pull outlet
Conveen Basic

Side action tap
Conveen Basic
Simplex S

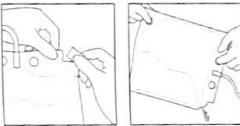

Without outlet tap
Conveen Basic
Simplex S

Removal

Disconnect the bedside drainage bag from the urethra or indwelling catheter.

Or, if the bedside drainage bag is linked to the leg bag, close the leg bag outlet tap. Then disconnect the bedside drainage bag. Avoid touching the open end of the leg bag outlet tube.

Intermittent catheterisation

After completing the catheterisation, disconnect the bedside drainage bag from the intermittent catheter.

Detach the bag from the hanger or stand.

Disposal

10



10. Change the bag according to local guidelines and recommendations.

The bag hanger or stand may be reused.

Empty the bag before discarding it.

The bag is intended for single use only and should be disposed of with normal household waste. Do not flush the product down the toilet.

Symbol



Do not use if package is damaged