**Boston  
Scientific**

# Urovac™ Bladder Evacuator

<b>Directions for Use</b>	<b>2</b>
<b>Instrucciones de uso</b>	<b>5</b>
<b>Mode d'emploi</b>	<b>8</b>
<b>Gebrauchsanweisung</b>	<b>11</b>
<b>Istruzioni per l'uso</b>	<b>14</b>
<b>Gebruiksaanwijzing</b>	<b>17</b>
<b>Instruções de Utilização</b>	<b>20</b>

90986319-01

2019-12

<b>TABLE OF CONTENTS</b>	
<b>WARNING</b> .....	<b>3</b>
<b>CONTENTS</b> .....	<b>3</b>
<b>INTENDED USE/INDICATIONS FOR USE</b> .....	<b>3</b>
<b>CONTRAINDICATIONS</b> .....	<b>3</b>
<b>PRECAUTIONS</b> .....	<b>3</b>
<b>ADVERSE EVENTS</b> .....	<b>3</b>
<b>HOW SUPPLIED</b> .....	<b>3</b>
Handling and Storage .....	<b>3</b>
<b>OPERATIONAL INSTRUCTIONS</b> .....	<b>3</b>
Preparation Before Use.....	<b>3</b>
<b>DIRECTIONS FOR USE</b> .....	<b>3</b>
To Fill Bottle.....	<b>3</b>
To Evacuate.....	<b>4</b>
Emptying and Disposal.....	<b>4</b>
<b>WARRANTY</b> .....	<b>4</b>

# Urovac™ Bladder Evacuator

## Rx ONLY

**Caution:** Federal Law (USA) restricts this device to sale by or on the order of a physician.

### WARNING

Contents supplied STERILE using an ethylene oxide (EO) process. Do not use if sterile barrier is damaged. If damage is found, call your Boston Scientific representative.

For single use only. Do not reuse, reprocess or resterilize. Reuse, reprocessing or resterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Reuse, reprocessing or resterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.

After use, dispose of product and packaging in accordance with hospital, administrative and/or local government policy.

### CONTENTS

The evacuator requires some assembly prior to use. The evacuator is comprised of:

- Clear PVC bottle with preattached cap and one-way filter
- Latex tubing
- Nozzle adaptor
- Cap

### INTENDED USE/INDICATIONS FOR USE

The Urovac Bladder Evacuator is intended to be used for evacuating irrigation solution containing surgical specimens, calculi or blood clots from the bladder.

### CONTRAINDICATIONS

Latex Allergy.

### PRECAUTIONS

Take care to assure that all fittings are hand tight before doing the procedure.

**Note:** This product contains DEHP (Bis(2-ethylhexyl) phthalate). BSC has conducted a Chemical Safety Assessment of DEHP in this device based on currently known acceptable exposure limits for sensitive populations (children and pregnant/nursing women), and has determined that the potential exposure is below the known acceptable exposure limits for the device when used in accordance with the indications and instructions for use.

**Caution:** This Product Contains Natural Rubber Latex Which May Cause Allergic Reactions.

### Caution:

- Use sterile technique.
- Use proper safety precautions for body fluid contact.

### ADVERSE EVENTS

None known.

### HOW SUPPLIED

The Urovac Bladder Evacuator is supplied sterile by ethylene oxide gas. Do not use if package is opened or damaged. Do not use if labeling is incomplete or illegible.

### Handling and Storage

Store in a cool, dry, dark place. Rotate inventory so that devices are used prior to the expiration date on package label.

### OPERATIONAL INSTRUCTIONS

#### Preparation Before Use

- Select the correct nozzle end which is sized for the ACMI, STORZ, Olympus or Wolf Resectoscope.
- Firmly insert other end of nozzle into rubber tubing and attach tubing to the cap spout.

### DIRECTIONS FOR USE

#### To Fill Bottle

- Hand tighten screw cap.
- Test for leaks by pressing a fingertip over the nozzle opening and then squeezing the bottle. If air leaks, then tighten the cap slightly, and check that the tubing is securely attached. If there is still a leak, do not use the evacuator.
- Invert the evacuator and place the nozzle into a basin of irrigation solution. Squeeze the bottle to expel air and relax pressure to fill bottle with irrigation solution. Repeat procedure until all air is expelled.

#### To Evacuate

- Connect the nozzle to the sheath of the resectoscope, press firmly inward.
- Fill the bladder with the appropriate volume of irrigation solution.
- Alternately, squeeze the bottle and then relax to suction the tissue. Continue until no more tissue enters the system. \*Note that tissue in the tubing and nozzle may be recycled into the bladder. Only the tissue in the bottle is prevented from re-entering the bladder.
- Sweep the resectoscope side-to-side while evacuating to clear the base of the bladder. Should one filling not be adequate to remove all tissue, the evacuator may be recharged by emptying and refilling with fresh irrigation solution.

#### Emptying and Disposal

- After completing the procedure, all of the tissue will be in the bottle. To drain the irrigation solution, invert the bottle and gently squeeze.
- To remove the tissue, unscrew the cap and transfer tissue to a suitable container.
- The tissue may be transported in the bottle. Remove the tubing from the cap spout and securely seal the spout with the supplied cap.

#### WARRANTY

Boston Scientific Corporation (BSC) warrants that reasonable care has been used in the design and manufacture of this instrument. **This warranty is in lieu of and excludes all other warranties not expressly set forth herein, whether express or implied by operation of law or otherwise, including, but not limited to, any implied warranties of merchantability or fitness for a particular purpose.** Handling, storage, cleaning and sterilization of this instrument as well as other factors relating to the patient, diagnosis, treatment, surgical procedures and other matters beyond BSC's control directly affect the instrument and the results obtained from its use. BSC's obligation under this warranty is limited to the repair or replacement of this instrument and BSC shall not be liable for any incidental or consequential loss, damage or expense directly or indirectly arising from the use of this instrument. BSC neither assumes, nor authorizes any other person to assume for it, any other or additional liability or responsibility in connection with this instrument. **BSC assumes no liability with respect to instruments reused, reprocessed or resterilized and makes no warranties, express or implied, including but not limited to merchantability or fitness for a particular purpose, with respect to such instruments.**

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All rights reserved.
Boston Scientific Limited
Boston Scientific (Australia) Pty Ltd
© 2019 Boston Scientific Corporation or its affiliates.