[MD]

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Cautery Tip Cleaner

Non-Sterile

[IC] [RSP]
 Aspen Surgical Products Inc. Ergo Europe
 6460 Southbelt Drive SE P.O. Box 200
 Columbus, OH 43230 2518 AP The Hague
 Phone: 614-698-7300 The Netherlands
 Toll Free: 800-364-7004
 aspensurgical.com Made in USA

English

Intended use:
Intended to assist in cleaning and removing material from caustery tips during surgical procedures.

Intended User/ Patient Target Groups:
Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Instructions for use:
Remove backing and apply to appropriate surface. Wipe caustery tip on pad to remove debris.

Precautions:
Device is single use only. Reuse of device could result in infection, contamination and/or device failure which could lead to patient harm. Device is intended for limited contact duration less than or equal to 30 hours.

Sterilization Instructions:
1. Place device into a tub or sterilization tray.
2. Using Ethylene Oxide sterilization method, sterilize according to the following parameters:

Cycle Parameter	Specification	Duration
Preconditioning		
Temperature	130T +/- 30T	
Humidity	65% +/- 15%	30.15 hrs - 72 hrs
Chamber		
Vacuum	27" +/- 3.0 mmHg	
Precondition Dwell	22.5" +/- 1.0 Hg	60 + 30.0 minutes
Gas Inject Dwell	18.2" +/- 0.5 Hg/hr	180 + 5.0 minutes
Soak Temperature	133T	
Aeration		
Temperature	126T	24 hrs - 72 hrs Max

Disposal:
Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Notice to Users and/or Patients in EU:
Any serious incident that has occurred in relation to the device, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.