


# NYLON TAPE

## – STERILE –



38907280

08/2019  
LAB1071419003

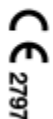

ar	NYLON TAPE شريط نايلون	et	NAILONLING	ko	나일론테이프	ru	НЕЙЛОНОВАЯ ЛЕНТА
bg	НАЙЛОНОВА ЛЕНТА	fi	NAILONNAUHA	it	NAILONO JUOSTELE	sk	NYLONOVÁ PÁSKA
cs	NYLONOVÁ PÁSKA	fr	BANDE NYLON	iv	NEILONA LENTE	sl	NAJLONSKI TRAK
da	NYLONTAPE	hr	NAJLONSKA VRPČA	nl	NYLON BAND	sv	NYLONBAND
de	NYLONBAND	hu	NYLON SZALAG	no	NYLONTAPE	tr	NAYLON BANT
el	ΝΑΙΛΟΝ ΤΑΪΝΙΑ	id	PLESTER NYLON	pl	TASMA NYLONOWA	zh-cn	尼龙带
en	NYLON TAPE	it	NASTRO DI NYLON	pt	FITA NYLON	zh-tw	尼龍膠帶
es	CINTA DE NYLON	kk	НЕЙЛОН БАУЫ	ro	BANDĂ DE NYLON		

### Instructions for use

#### NYLON TAPE STERILE

#### DESCRIPTION

Nylon Tape is a white (and/yed) flat woven or braided Nylon-(Polyamide 6,6) tape. Nylon Tape is available in various widths and lengths, non-needled only.

#### PERFORMANCE

Because of its flat woven structure, Nylon Tape is indicated for temporary ligation or retraction of tissues, organs or other anatomical structures during surgical procedures without injuring the tissue involved.

#### INDICATIONS

Nylon Tape is indicated for temporary ligation or retraction of tissues, organs or other anatomical structures during surgical procedures. The device is not intended to be implanted in the body.

#### APPLICATION

Nylon Tape is used for temporary retraction and/or fixing of organs or parts of organs (e.g. intestine) during an operation. After completion of the operation the tape should be removed and disposed of.

#### CONTRAINDICATIONS

There are no known contraindications. However, Nylon Tape is not indicated for implantation.

#### WARNINGS / PRECAUTIONS

Care should be taken during tissue retraction to avoid injury. Nylon Tape is provided as a sterile product. Do not resterilize. Do not use if packaging is opened or damaged. Discard opened unused products.

Reuse of this device (or portions of this device) may create a risk of product degradation and cross-contamination, which may lead to failure of treatment, infection or transmission of bloodborne pathogens to patients and users.

Safety and effectiveness of Nylon Tape have not been established for use as an implanted device. The risks associated with product implantation have not been established.

#### ADVERSE REACTIONS

None known.

en

#### STERILITY

Nylon Tape is sterilized by irradiation. Do not resterilize! Do not use if package is opened or damaged! Discard open, unused product.

#### STORAGE

No special storage conditions required. Do not use after expiry date!

#### SYMBOLS USED ON LABELING



Sterilized using irradiation



Do not reuse



Do not use if package is damaged



Do not resterilize



Batch code



Use by - year and month



CE-mark and Identification Number of Notified Body. The product meets the essential requirements of Medical Device Directive 93/42/EEC.



Manufacturer



Caution: See instructions for use



Authorized representative in the European Community



Number of Sachets in Box



Catalogue Number