

Instructions for Use for BARDIA™ Bed Bags

Contains or Presence of Phthalates: Di(2-ethylhexyl) phthalate (DEHP) is a plasticizer used in some polyvinyl chloride medical devices. DEHP has been shown to produce a range of adverse effects in experimental animals, notably liver toxicity and testicular atrophy. Although the toxic and carcinogenic effects of DEHP have been well established in experimental animals, the ability of this compound to produce adverse effects in humans is controversial. There is no evidence that neonates, infants, pregnant and breast feeding women exposed to DEHP experience any related adverse effects. However, a lack of evidence of causation between DEHP-PVC and any disease or adverse effect does not mean that there are no risks.

A space has been provided for you to write the date on which the urine bag should be changed. Please follow local policy regarding how long the bag can remain in place.

This is a single use device. Do not resterilize any portion of this device. Reuse and/or repackaging may create a risk of patient or user infection, compromise the structural integrity and/or essential material and design characteristics of the device, which may lead to device failure, and/or lead to injury, illness or death of the patient.



Use By
 À utiliser avant
 Verwendbar bis
 Da utilizzarsi entro
 Utilizar antes de
 Uiterste gebruiksdatum
 Usar antes de
 Ημερομηνία λήξης
 Sidste anvendelsesdato
 Används före
 Käytä ennen
 Brukes innen
 Zużyć przed
 Felhasználható
 Použitelné do
 Son kulanım tarihi
 Срок годности
 使用期限
 사용 기한
 Срок на годност
 Upotrijebiti do
 Kõlblikusaeg
 Izljetot līdz
 Naudoti iki
 Utilizati până la
 Použite do
 Uporabno do
 Notist fyrr



Catalog Number
 Numéro de catalogue
 Katalog-Nummer
 Numero di catalogo
 Número de catálogo
 Catalogusnummer
 Referência de catálogo
 Αριθμός καταλόγου
 Katalognummer
 Katalognummer
 Luettelonumero
 Katalognummer
 Numer katalogowy
 Katalogszám
 Katalogové číslo
 Katalog numarası
 Каталогnummer
 Каталог номер
 Kataloški broj
 Kataloginumber
 Kataloga numurs
 Katalogo numens
 Număr de catalog
 Katalogové číslo
 Kataloška številka
 Vörulstanúmer



Bard and Bardia are trademarks and/or registered trademarks of C. R. Bard, Inc.
 Bard et Bardia sont des marques et (ou) des marques déposées de C. R. Bard, Inc.
 Bard und Bardia sind Marken und/oder eingetragene Marken von C. R. Bard, Inc.
 Bard e Bardia sono marchi commerciali e/o registrati di C. R. Bard, Inc.
 Bard y Bardia son marcas comerciales y/o comerciales registradas de C. R. Bard, Inc.
 Bard en Bardia zijn handelsmerken en/of gedeponeerde handelsmerken van C. R. Bard, Inc.
 Bard e Bardia são marcas comerciais e/ou marcas registradas da C. R. Bard, Inc.
 Οι ονομασίες Bard και Bardia είναι εμπορικά σήματα ή/και εμπορικά σήματα κατατεθέντα της C. R. Bard, Inc.
 Bard og Bardia er varemærker og/eller registrerede varemærker tilhørende C. R. Bard, Inc.
 Bard och Bardia är varumärken/inregistrerade varumärken som tillhör C. R. Bard, Inc.
 Bard ja Bardia ovat C. R. Bard, Inc:n tavaramerkkejä ja/tai rekisteröityjä tavaramerkkejä.
 Bard og Bardia er varemerker og/eller registrerte varemerker som tilhører C. R. Bard, Inc.
 Bard i Bardia są znakami towarowymi i/lub zastrzeżonymi znakami towarowym firmy C. R. Bard, Inc.
 A Bard és a Bardia a C. R. Bard, Inc. védjegye és/vagy bejegyzett védjegye.
 Bard a Bardia jsou ochranné známky a/nebo registrované ochranné známky společnosti C. R. Bard, Inc.
 Bard ve Bardia, C. R. Bard, Inc. şirketinin ticari markaları ve/veya tescilli ticari markalarıdır.
 Bard и Bardia являются товарными знаками и (или) зарегистрированными товарными знаками компании C. R. Bard, Inc.
 Bard 和 Bardia 為 C. R. Bard, Inc. 的商標和/或註冊商標。
 Bard 및 Bardia는 C. R. Bard, Inc.의 상표 및/또는 등록 상표입니다.
 Bard и Bardia са търговски марки и/или регистрирани търговски марки на C. R. Bard, Inc.
 Bard i Bardia zaštitni su znakovi i/ili registrirani zaštitni znakovi društva C. R. Bard, Inc.
 Bard ja Bardia on ettevõtte C. R. Bard, Inc. kaubamärgid ja/või registreeritud kaubamärgid.
 Bard un Bardia ir uzņēmuma C. R. Bard, Inc. reģistrētas preču zīmes.
 „Bard” ir „Bardia” yra bendrovės „C. R. Bard, Inc.” prekių ženklai ir (arba) registruotieji prekių ženklai.
 Bard și Bardia sunt mărci comerciale și/sau mărci comerciale înregistrate ale C. R. Bard, Inc.
 Bard a Bardia sú ochranné známky a/alebo registrované ochranné známky spoločnosti C. R. Bard, Inc.
 Bard in Bardia sta blagovni znamki in/ali registrirani blagovni znamki družbe C. R. Bard, Inc.
 Bard og Bardia eru vörumerki og/eda skrásett vörumerki C. R. Bard, Inc.



Manufacturer
 Fabricant
 Hersteller
 Fabbricante
 Fabricante
 Fabrikant
 Fabricante
 Κατασκευαστής
 Producent
 Tilverkare
 Valmistaja
 Tivriker
 Producent
 Gyártó
 Výrobce
 Uretis
 Производител
 製造商
 제조업체
 Производител
 Proizvođač
 Tootja
 Ražotājs
 Gaminiojas
 Producător
 Výrobca
 Proizvajalec
 Framleiðandi



Do Not Resterilize
 Ne pas restériliser
 Nicht resterilisieren
 Non risterilizzare
 No reesterilizar
 Niet hersteriliseren
 Não reesterilizar
 Μην επαναστεριώνετε
 Må ikke resteriliseres
 Får ej omsteriliseres
 Ei saa steriloida uudestaan
 Må ikke resteriliseres
 Nie sterylizować ponownie
 Ne sterilizálja újra!
 Nesterilizujte opakovaně.
 Yemiden Sterilize Etmeyin
 Не подвергать повторной стерилизации
 請勿重複滅菌
 재소독하지 마십시오
 Да не се стерилизира повторно
 Ne sterilizirati ponovno
 Mitte sterilsendada uuesti
 Aikartoli nesterilizēt
 Nesterilizuoti pakartotinai
 Nu resterilizați
 Opätovne nesterilizujte
 Ni za ponovno sterilizacija
 Endursæfð ekki



Sterilized by ethylene oxide
 Stérilisé à l'oxyde d'éthylène
 Sterilisiert mit Ethylenoxid
 Sterilizzato con ossido di etilene
 Esterilizado con óxido de etileno
 Gesteriliseerd met ethyleenoxide
 Esterilizado com Óxido de Etileno
 Αποστειρωμένο με οξείδιο του αιθυλενίου
 Steriliseret med etylenoxid
 Steriliserad med etylenoxid
 Steriloitu etyleenoksidilla
 Sterilisiert med etylenoksid
 Sterylizowany tlenkiem etylenu
 Etilén-oxidál sterilizálva
 Sterilizováno ethylenoxidem
 Etílen-oksíde sterilize edilmisdir
 Стерилизовано этиленоксидом
 以環氧乙烷滅菌。
 산화에틸렌으로 멸균 처리됨
 Стерилизирано с этиленов оксид
 Sterilizirano etílen-oksídom
 Steriliseeritud etüleenoksidiga
 Sterilizēts ar etilēnoksidu
 Sterilizat cu oxid de etilenă
 Sterilizované etýlenoxidom
 Sterilizirano z etílenoksidom
 Sæft með etýlenoxíði



Manufacturer:
 C. R. Bard, Inc.
 Covington, GA 30014 USA
 1-800-526-4455
 www.bardmedical.com



BD Switzerland Sàrl
 Terre Bonne Park – A4
 Route de Crassier 17
 1262 Eysins, Switzerland

