




# Surgical Skin Markers

REF	SIZE OF TIP	CAP COLOUR	QTY PER CARTON
VX1100	Bold Line Skin Marker		50
VX110	Bold Line Skin Marker with Ruler		50
VX110/5	Bold Line Skin Marker with Ruler (100x5 Units)		500
VX200	Fine Line Skin Marker		50
VX210	Fine Line Skin Marker with Ruler		50
VX210/5	Fine Line Skin Marker with Ruler (100x5 Units)		500
VX5100	Broad Line Skin Marker		50
VX5110	Broad Line Skin Marker with Ruler		50
VX5110/5	Broad Line Skin Marker with Ruler (100x5 Units)		500

 **Viomedex Ltd**, Unit 13 Swan Barn Business Centre, Hailsham, East Sussex, BN27 2BY,  
 T:+44 (0) 1323 446 130, F:+44 (0) 1323 847 417, E: [sales@viomedex.com](mailto:sales@viomedex.com), W: [www.viomedex.com](http://www.viomedex.com)



• Choice of Three nibs for better performance

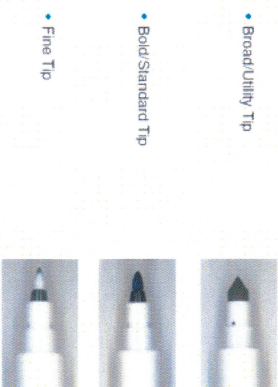

• Broad/Utility Tip

• Bold/Standard Tip

• Fine Tip

CE 1282

### en

#### Indications for Use

1. Cleanse patient's skin with alcohol or soap and water. Allow area to dry thoroughly.
2. Mark incision locations as necessary. Allow ink to dry thoroughly before prepping.
3. Prepare skin with Povidone-iodine (Betadine) solution to disinfect. Ink markings may fade slightly but remain legible.
4. Markings can be removed with soap and water. Markings will gradually disappear with soap & water.

#### Warnings

1. For single patient use only.
2. Dispose of as per normal hospital procedures
3. Do not re-sterilize.
4. Skin surface must be clean and dry before marking skin.
5. Consider whether the patient is sensitive to Gentian Violet.
6. Please dispose as per normal hospital procedures if the expiry date has been passed.



Do not re-use



Sterilized Using Irradiation



Keep Dry



Keep Away from Sunlight



Manufacturer



Do Not Resterilize



Latex Free



Do Not use if Package is Damaged