



## Before using an oral swab...

Please check the head is firmly attached to the paper stick.


Rociale Healthcare is committed to promoting patient safety and welfare. Any object placed into the mouth could present a choking hazard. Therefore as an added precaution please follow the instructions below carefully.

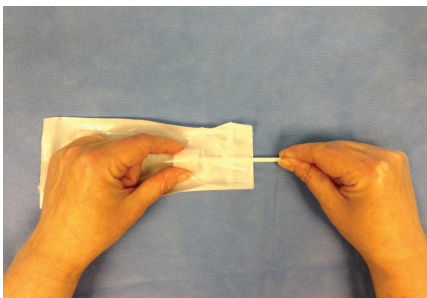

**Grip the foam head between the packaging and gently tug on the stick.**

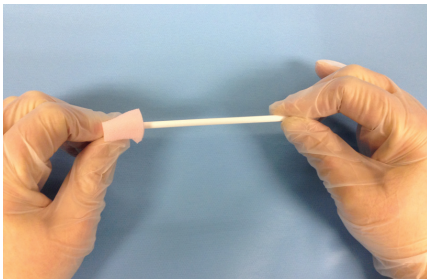


**Alternatively, use a sterile gloved hand and check the foam head is attached to the stick.**

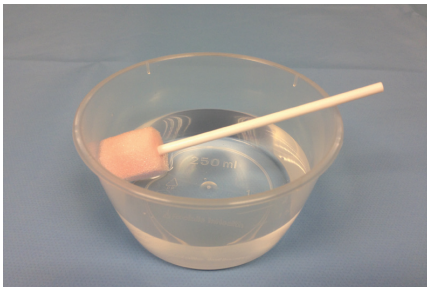

**DO NOT** allow the oral swab to soak in liquid as the foam head is liable to become detached.

**After use, discard the swab immediately.**

If you break down and distribute these packs, please ensure this information is passed on.

 Finess Healthcare Group Co., Ltd, No. 1503 A1 Block , No. 203 Lantian Road, 315000 Ningbo, PEOPLE ' S REPUBLIC OF CHINA

 ZOUSTECH S. L., Pso. Castellana, 141-Planta 19, 28046 Madrid, SPAIN

Call: 01443 471300

E-mail: [info@rocialehealthcare.com](mailto:info@rocialehealthcare.com)

Web: [www.rocialehealthcare.com](http://www.rocialehealthcare.com)

