

SINGLE-USE MEDICAL INSTRUMENTS

Instructions for Use

All products are sterile, latex-free, single-use instruments intended for use in natural body cavities. They should be used only by suitably trained staff in a clean environment.

All instruments have a unique product identification number which can be used on patient's notes for traceability.

- The device should remain in its original protective pouch and carton until use, and should be stored in a cool, dry place.
- Open the sterile peel pouch, remove instrument, then carry out the examination or procedure. Do no use if packaging is damaged.
- Products are sterile and single-use, therefore should not be washed or sterilised for re-use.
- The expiry date is printed on each pouch. Usage after this date could affect the sterility of the device.
- All clinical waste should be disposed of in accordance with Local Health Authority and Department of health Guidelines.

This device is intended for an individual patient, during a single procedure and should be discarded after use. Potential of cross contamination if re-used due to device not being sterile.

Product	Size	Pack Size	Product Code	Size	Pack Size	Product Code
Sponge holder	-	50	400300	-	5	400302
Uterine Sound	-	100	400310	-	10	400311
Coil Remover	-	50	400317	-	5	400318
Long Scissors	-	50	400320	-	5	400321
Tenaculum	-	50	400330	-	5	400331
Vulsellum	-	50	400335	-	5	400336
Dilator	3/4mm	100	400342	3/4mm	10	400343
Dilator	5/6mm	100	400344	5/6mm	10	400345
Dilator	7/8mm	100	400347	7/8mm	10	400348
Thread Retriever	-	5	400560	-	-	-
PELLjelly Lubricant	-	100	400400	-	200	400400
PELLpack IUD removal & Fitting kit	Medium PELLspec	4	400351	Medium Long PELLspec	4	400352
PELLpack IUD removal kit	Medium PELLspec	4	400361	Medium Long PELLspec	4	400362
Proctoscope	Paediatric	25	400370	Medium	25	400380
Proctoscope	Large	25	400390	-	-	-

Williams Medical Supplies Ltd
The Maerdy Industrial Estate North
Rhymer, Gwent, NP22 5PY
sales@wms.co.uk
www.wms.co.uk
Tel: 01685 846666




DCC Vital
Fannin House, South County
Business Park, Leopardstown,
Dublin 18, Ireland



IFU5360119


DILATOR

- Hold the Dilator in the middle at the thumb hold (see diagram opposite).
- Choose which size end you wish to use - the sizes are engraved either side of the thumb hold.
- For coil fitting it advisable to use the smaller end first.
- Both ends are tapered to allow the Dilator to pass smoothly and comfortably into the uterine cavity. Force should not be applied when dilating the cervix as this could cause serious injury.

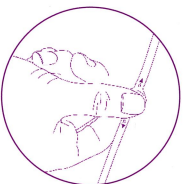

COIL REMOVER

- Visualise the cervix and coil threads then note their position. Introduce the Coil Remover into the speculum in a closed position ensuring the tip of the “V” is pointing upwards (see diagram opposite).
- Locate the coil threads and slowly open the device.
- The coil threads must be trapped in the “V” of the Coil Remover (see diagram opposite) and the device must be locked securely at the base of the instrument.
- Once the thread is trapped and the device locked, gently withdraw. Do not pull too hard or fast on the threads as they could slip and the procedure will need to be repeated.

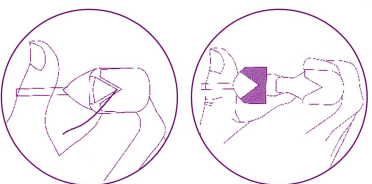

THREAD RETRIEVER


- Carefully insert the Thread Retriever into the uterine cavity.
- Thoroughly but softly “sound” the uterus to determine the presence or absence of an IUD.
- If an IUD is located, rotate the Thread Retriever and with gentle pressure sweep the posterior wall and at the same time gradually and slowly withdraw the Thread Retriever. In this way the threads may be picked up and brought into view at the external os.
- The procedure can be repeated and the anterior and lateral walls can be swept.
- When the threads have been brought into view the IUD may be removed or left in situ for further follow up.

UTERINE SOUND

- Gently insert the Uterine Sound to determine the depth and direction of the uterine cavity. It may be necessary to grasp the cervix with a Tenaculum or Vulsellum to align the cervical canal with the uterine cavity.
- Apply gentle traction to straighten any cervical curvature.
- With raw the Uterine Sound. The length from the interior fundus to the external cervical os can be measured directly using graduations indicated on the body of the sound.

LONG SCISSORS

- Following the successful IUD fitting, introduce the Long Scissors through the speculum.
- Cut the coil thread. Use the tip of the blade to cut the thread (see diagram opposite).
- Do not pull the thread.
- When the coil thread has been trimmed, gently withdraw the Long Scissors.

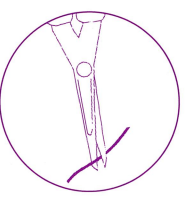

PROCTOSCOPE

- Lubricate the Proctoscope with a water based lubricant such as PELJelly.
- Gently introduce the Proctoscope into the anal canal. Never introduce the Proctoscope without its obturator present.
- When the Proctoscope has been completely introduced, remove the obturator and perform the examination.
- The rest of the anal canal can be examined while the Proctoscope is being gently withdrawn.