

Irrigation Set Instructions for Use

CAUTION

General

- Device must be used by trained clinician.
- Do NOT use if product is damaged.
- Single use only - reuse compromises safety and efficacy.
- Do NOT resterilise.
- Cumulative use exceeding 30 days is NOT recommended.
- Ensure all connections are secure before priming the set. Do NOT over-tighten. Do NOT use any instruments to tighten connections.
- Ensure all air is expelled from the device before connecting to patient. Do NOT allow air to enter the set during use.
- Clamp all fluid lines (including side-arms) when not in use.
- Dispose per hospital protocol or as clinical waste.


Irrigation Set

- Replace set per hospital protocol. Replacement interval exceeding 72 hrs is NOT recommended.

INSTRUCTIONS FOR USE

Using Luer Connectors

1. Use only standard IV Luer fittings for Luer connections.
2. Use of Luer lock connections is recommended.
3. Where product features a male Luer lock with rotating collar, insert the Luer with a firm push and turn motion before engaging the rotating collar.
4. Cap all Luer connectors when not in use with sterile end cap.

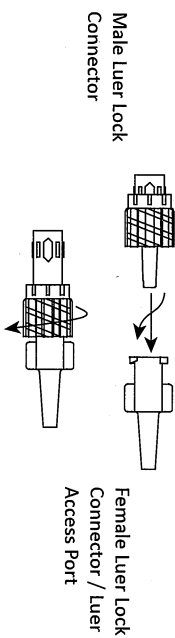

INSTRUCTIONS FOR USE

Irrigation Set

1. Close the flow regulator (roller regulator or clamp).
2. Remove cap from the perforator and connect to the fluid source.
 - Perforate the container at the designated port only.
3. Prime the set: use the flow regulator to fill the line with fluid. Ensure all air is expelled from the device before connecting to downstream device.

STERILE EO

