English

DESCRIPTION OF THE PRODUCT

Yyclic is a lipid-resistant 3-way stopcock with one male Luer-lock port and two female Luer-lock ports. All Luer-lock ports are protected by a cap to prevent touch contamination after opening the blister pack. Yyclic can be used with any type of patient.

The Yyclic stopcock is supplied with:

- a safety click so the user can feel the chosen position of the key
- a mobile safety lock to prevent accidental disconnections
- safety identification caps (for some references) for indicating venous use (blue) or arterial use (red).

This device is not made with dry or natural rubber latex.

OPERATING PRECAUTIONS

- The practitioner/nurse using the device must know and apply all the standard precautions to prevent complications and problems related to the use of 3-way stopcocks. Use an aseptic technique to prevent infections.
- Check the integrity of the sterile packaging before use.
- Do not over-tighten the connections.
- Regularly check the connections between the 3-way stopcock and the connected medical devices.
- Replace the stopcocks according to current recommendations and/or the internal procedures implemented by your company.
- To dispose of, place contaminated items in the appropriate receptacle.
- Re-use of this device may change its mechanical or biological features, and may cause device failure, allergic reactions or bacterial infections.
- Never connect Yyclic onto another Yyclic.

PROTOCOL FOR USE

1. Unscrew the male Luer protector on the stopcock (fig 1).
2. To connect Yyclic to a female Luer, hold the safety lock on the stopcock (fig 2).
3. Connect the male Luer to the female Luer (fig 3).
4. Keep the female Luer immobilised while tightening the safety lock with the other hand (fig 4). **CAUTION:** Tighten the mobile safety lock until obvious resistance can be felt.
5. To check the connection, hold the body of the stopcock (fig 5) and turn the stopcock the opposite way to the tightening direction (fig 6).

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