

	INSTRUCTIONS FOR USE	
	DISSECTORS AND ELEVATORS	TF01-IFU-03
	REVISION 1	

Intended use

Elevators and Dissectors are manual surgical instruments used to elevate, dissect, and remove fragments of tissue and or bone. They are available in varying lengths and widths.

Reuse of Single Use Product

Product is designated as 'Single Use' and must not be reused. It should only be used on an individual patient during a single procedure and then discarded. It is not intended to be reprocessed and used again, even on the same patient. Non hazardous use cannot be guaranteed if these products are used again, since there is a risk of infection and or the safety of the product is not longer assured.

Product inspection before use

Inspect device for rust, dents, cracks, pits and pores before use.

Warning

- Do not use if product or sterile barrier system is damaged.
- Products supplied non sterile must be sterilised before use.
- Please follow our sterilisation instructions prior to use.
- Do not use beyond the expiry date stated on the packaging.

Sterilisation

Instruments can be sterilised using the following methods:

Ethylene Oxide

Concentration: up to 450mg/l

Temperature: 60°C

Exposure: up to 6 hours

Humidity: up to 100%

Irradiation

Dose to be validated by user, up to 70kGy

Contraindications

Instruments should not be used for anything other than their intended use. Not to be used on patients who have an allergy or sensitisation to stainless steel.

Intended User

Our instruments are designed for use by appropriate qualified clinicians, who will have good knowledge of their features and how they should be used. These products are not intended for use by lay persons.

Disposal:

Used instruments should be disposed of in accordance with local clinical waste guidance.

		 Rociale Healthcare Ltd, Ty Mynydd, Cwm Cynon Business Park (North), Mountain Ash, Rhondda Cynon Taff, CF45 4ER, UK	<table border="1"> <tr> <td>EC</td> <td>REP</td> </tr> </table> Advena Limited, Tower Business Centre, 2 nd Flr, Tower Street, Swatar, BKR 4013, Malta	EC	REP
EC	REP				