

Instructions For Use

Universal Aseptic Pack

Open pack onto the clinically clean work surface

Wash hands with soap and water

Carefully open outer wrap onto the work surface creating a sterile field

Put on the provided apron

Keeping your hand within the disposal bag remove any existing dressings from the wound

Invert the dressing within the bag and then secure the waste bag in an easily accessible location

Wash hands according to local policy

Apply sterile gloves from the pack


Cleanse and dress wound according to local policy

Once the procedure is complete fold the sterile field inwards and place the contents in the waste bag

Seal waste bag and dispose of as per policy

 Rociale Healthcare Ltd, Cwm Cynon Business Park (North),
Mountain Ash, Rhondda Cynon Taf, South Wales, CF45 4ER, UK

 Xuchang Zhende Surgical Dressing Co.,Ltd.
Area A, Baihua Road East, South Side of Erji river, Industry
Cluster District, Yanling County, 461200 Xuchang, Henan, China.
www.zhende.com

 Shanghai International Holding Corp. GmbH(Europe)
Eiffestraße 80, 20537 Hamburg GERMANY



Download the IFU from below website:
www.rocialehealthcare.com/ifu-downloads/



SEE PACKAGING

Date of issue: 2022-11-08
Version number: 01