# PENCIL AND ELECTRODES FOR ELECTROSURGERY

## INFORMATION FOR SAFETY AND PERFORMANCES

### INTENDED USE

Cutting and coagulation of tissues during electrosurgical procedures, with the use of a compatible high frequency generator.

### CONTRAINDICATIONS

The products must not be utilised if:

- there is visible damage to the electrode or extension, to its insulation, to the body and/or connector of the pencil, or to the connection cable,
- the preliminary operational check gave negative results,
- the electrode or extension is not firmly inserted in the distal part of the product.

**NB:** pacemakers may be damaged by electrosurgical current. Do not submit a patient wearing a pacemaker to electrosurgical current without first consulting a cardiologist.

### PRECAUTIONS AND WARNINGS

- STERILISE BEFORE USE those products which are not pre-sterilised.
- After controlling the packing for damages, check the hand-switch pencil, electrode, cable, and connector. Do not use the product in the case of damages or visible defects. Instead, return it to FIAB.
- The distal part of the hand-switch pencil can hold all models of electrodes and extensions having a shank diameter of 2.38 mm (3/32") or 4 mm depending on the model. Be sure that the electrode or extension (if present) is firmly inserted in the pencil, and that the insulating coupling of the electrode and extension (if present) is intact.
- Always check the functioning before use. For the hand-switch models, make sure that the two push-buttons make a "click" when they are pressed. Discard defective products.
- Be sure that the connection of the hand-switch pencil's plug with the generator is firm, and that the metallic part of the connector is completely inserted.
- Do not use clips to secure the cable of the pencil to the surgical cloth, in order to avoid damage to the cable.
- Keep the products away from inflammable materials: their accidental activation and heat could cause fires.
- When the products are not in use, they should be kept in suitable containers.
- High-frequency electrosurgical generators, the grounding plates and accessories to be used with the present product, should conform to existing regulations.
- Refer to the instructions for use supplied with the product regarding use of the generator and the grounding plates.
- The devices are to be used by trained healthcare professionals in electrosurgical procedures.

*MAXIMUM APPLICABLE VOLTAGE 9kVpp*

## **POSSIBLE CAUSES OF BURNS**

Burns may be caused by a high density of current in the patient's tissues or by the heating of inflammable fluids or gases. Possible causes are:

- the patient has inadvertently been placed in contact with electrically-conductive parts,
- a direct contact has occurred between the cables and the patient's skin, which has caused a capacitive effect,
- the combustion of inflammable disinfectant agents,
- the combustion of inflammable narcotic gases.

## **VALID ONLY FOR DISPOSABLE PENCIL**

The product is disposable and sterilised with EO gas. Do not reuse. Reusing the product may lead to:

- possible presence of biological residues that may cause cross-infections,
- alteration of materials,
- loss of initial functional features of product.

Sterility is not guaranteed if the package has been opened and/or damaged.

Do not use after the expiration date. Sterility is not guaranteed.

## **VALID ONLY FOR REUSABLE PENCIL**

**CLEANING** - Disconnect the pencil from the generator. Remove the electrode and extension (if present) from the tip of the pencil. To clean the products, it is advisable to use decontaminating solutions and proteolytic detergents for surgical instruments. While cleaning the electrodes, the operator should take care not to wound him or herself with the blades or tips. Follow the relative instructions when using these products.

After cleaning, rinse with water and keep the pencil in an upside-down position for at least 30 minutes, so as to enable the exit of any liquids from the tip.

- Do not press the push-buttons during the washing operation.
- Do not make forced entry of liquids or air inside the receptacle of the tip.
- Do not use ultrasound baths.
- The use of mechanical washing systems may reduce the product's life.
- Do not immerse the pencil in any sort of solution during the 4 hours following a pre-vacuum treatment.
- Do not use abrasive mechanical systems on the insulation of the electrode or extension to remove biological residue, as such systems could damage it.

**STERILIZATION** - The devices in object of present instructions can be sterilised with ethylene oxide (ETO) or in an autoclave.

**NB:** The sterilisation process used must conform with existing regulations.

- Keep a record concerning checks carried out and the results obtained.
- Validation of the process is the responsibility of the user.
- Do not sterilise the pencil when the electrode or extension is inserted in the tip.
- Remove the protective cap from the electrode before autoclave sterilization. After the first use, the treatment cycle of the electrode (decontamination, sterilization) prior to re-use does not require the re-insertion of the protective cap, which can therefore be discarded.
- In repackaging, use adequate protection systems for the tip in order to avoid perforation of the pouch, with the consequent loss of sterility.

For sterilisation with ethylene oxide, the clean products must be packaged in suitable gas-proof pouches. An adequate ventilation time should be respected to ensure total dispersion of the ETO residues.

For sterilisation in an autoclave, package the products in suitable pouches.

Avoid wrapping that is too tight and sharply twisting of the cord.  
Advisable temperatures and times for steam autoclave are as follows:

<b>Cycle</b>	<b>Pre-vacuum</b>	<b>Exposure</b>	<b>Drying</b>
134°C (273°F)	8 min	134°C (273°F) 2,05 bar for 12 min	9 min
121°C (250°F)	8 min	121°C (250°F) 1,05 bar for 20 min	9 min

**NB:** The pencils can NOT be sterilised by 137°C (279 °F) cycle.

### **ATTENTION**

Should the products come in contact with the metal supports within the autoclave, their life may be shortened. It is therefore necessary the use of appropriate means spacing or wrapping as gauze.

Via Costoli, 4 - 50039 - Vicchio  
(Florence, Italy) - [www.fiab.it](http://www.fiab.it)

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