PT

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
STERILE EO Rx Only (USA)


Becton, Dickinson and Company, 1 Becton Drive, Franklin Lakes, NJ 07417-1885 USA
bd.com/vacutainer
© 2017 BD, BD, the BD Logo and all other trademarks are property of Becton, Dickinson and Company.

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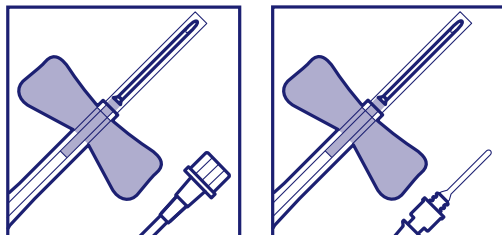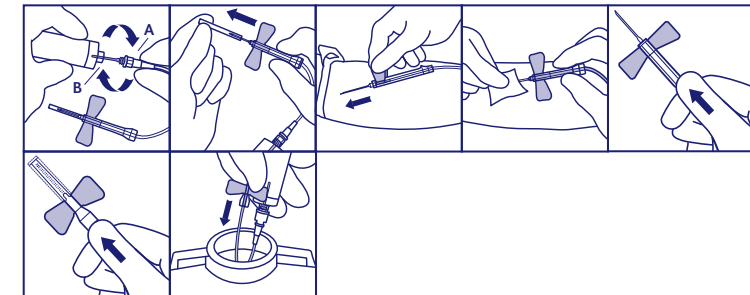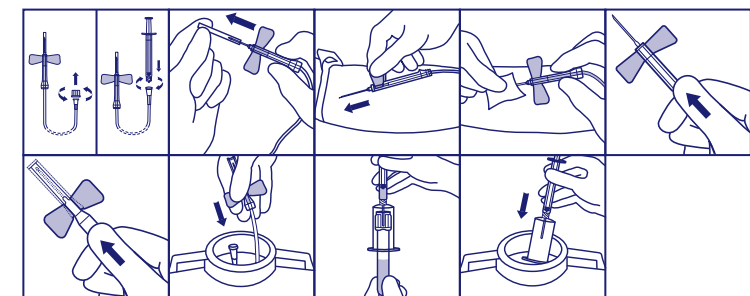
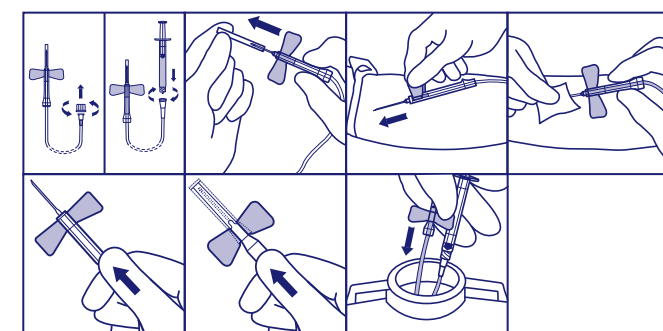
EN

Note: Venipuncture should be performed wearing gloves.

For Blood Collection

- 1. Check to ensure that the Female Luer Adapter (A) is securely attached to the Male Luer Adapter (B) and that the Male Luer Adapter (B) is securely attached on the Holder.
2. Remove needle sheath.
3. Perform venipuncture by holding wings.
4. Withdrawing Safety-Lok™ Blood Collection Sets: Apply pressure to site using your fingers as shown.
• Grasp the translucent yellow safety shield grip area with the thumb and index finger while at the same time grasping the tube.
5. Safety-Lok™ Shield Activation:
• One-hand Technique: Hold tubing in hand and advance translucent yellow safety shield with thumb and index finger until a click is heard indicating that the safety shield is locked in place over needle tip.
Modified One-Hand Technique Withdrawal Activation: Withdraw blood collection set by grasping the translucent yellow safety shield grip area with the thumb and index finger. With opposite hand, grasp tubing between thumb and index finger while pushing the yellow safety shield forward until a click is heard indicating that the needle is completely retracted and the safety shield is locked in place.
6. Dispose of all used materials in an appropriate container and wash hands.

For Blood Collection Using a Syringe

- 1. Remove luer cap and attach syringe. Follow Blood Collection steps 2 – 6.
Note: Use the BD Vacutainer® Blood Transfer Device to transfer blood from the syringe to the evacuated tube.

For Short Term IV Administration (up to 2 hours)

- 1. Remove luer cap.
2. Attach IV line or syringe to the Female Luer Adapter. Remove needle shield.
3. Prime set in accordance with recommended procedure.
4. Perform venipuncture by holding wings. Administer medication. • It is recommended that this device be changed per the institution's procedures, but no longer than 2 hours.

5. Safety-Lok™ Shield Activation: See Section 5 under Blood Collection (above)

- 6. Dispose of all used materials in appropriate container and wash hands.

UNIVERSAL PRECAUTIONS

Handle all biologic samples and blood collection "sharps" (lancets, needles, luer adapters, and blood collection sets) in accordance with the policies and procedures of your facility. Obtain appropriate medical attention in the event of any exposure to biologic samples (e.g. through a puncture injury) since samples may transmit viral hepatitis, HIV (AIDS), or other infectious diseases. Utilize any built-in used needle protector if the blood collection device provides one. Discard all blood collection "sharps" in biohazard containers approved for their disposal.

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