## OPSITE®

Transparent adhesive film

**Description**

OPSITE® Transparent adhesive film is a sterile, transparent polyurethane film with acrylic adhesive. The film is moisture vapour permeable, conformable and extensible. The film acts as a bacterial barrier which may help minimise the risk of infection. Two handles are located at the edges of the dressing to aid application. Individual dressings are packaged in sealed pouches and sterilised.

**Mode of Action**

OPSITE Film is highly moisture vapour permeable. It is waterproof, breathable and acts as a bacterial barrier on the skin. The acrylic adhesive on the film and the green handles provide secure fixation to the skin.

**Indications**

OPSITE Film is indicated for:

- Use as an incision drape in surgical procedures
- Superficial wounds (e.g. minor burns, scalds, abrasions, and lacerations)
- Split-thickness skin graft donor sites
- Closed surgical wounds
- Fixation of devices to the skin (i.e. catheters, tubing, primary dressings)

**Precautions for use**

If the wound becomes infected, infection should be treated as per local clinical protocol. In common with all adhesive products some cases of irritation and/or maceration of the skin surrounding the wound have been reported. As with all adhesive products, OPSITE Film should be applied and removed carefully from sensitive and fragile skin. Frequent dressing changes, particularly on patients with fragile skin or dermatological conditions, can result in skin reddening or damage to the skin. Should any signs of irritation, maceration, hypergranulation or sensitisation appear, discontinue use. Once opened, do not retain unused dressing for application at a later date.

**Instructions for use**

- Clean and dry the application area as per local clinical protocol.
- Remove the middle backing paper.
- Apply the dressing and smooth down (for the larger sized dressing, more than one Clinician may be required to apply the dressing).
- Remove the remaining two handles and smooth down.
- To remove the dressing, lift one corner and gently stretch parallel to the skin.

The dressing may be left in place for up to 7 days.

Dispose of according to local clinical protocol.

If during use of this device you believe that a serious incident has occurred, please report it to the manufacturer and your competent authority.
complaints@smith-nephew.com

<b>Product Availability</b>					
<b>Code</b>	<b>Size</b>	<b>4in x 5½in.</b>	<b>Carton</b>		
4963	10cm x 14cm / 4in. x 5½in.		10		
4975	10cm x 14cm / 4in. x 5½in.		50		
4542	10cm x 28cm / 4in. x 11in.		10		
4986	15cm x 28cm / 6in. x 11in.		10		
4967	25cm x 14cm / 10in. x 5½in.		20		
4987	30cm x 28cm / 11¾in. x 11in.		10		
4995	40cm x 42cm / 15¾in. x 16½in.		10		
4988	45cm x 28cm / 17¾in. x 11in.		10		
4989	45cm x 55cm / 17¾in. x 21¾in.		10		
4994	56cm x 84cm / 22¼in. x 33¼in.		10		

\*Trade Mark of Smith & Nephew
Date of issue 06/2022. Revision 01
This product may be covered by one or more US patents. www.smith-nephew.com/patents

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